

Micor point mole freckle removal pen



Introuduction :

Plasma Freckle removal pen is new product in 2015, easy to take.

Plasma Freckle removal pen work principle : similar as carbon dioxide (CO₂) laser beauty machine, it use new generation of plasma micro district discharge technology with low temperature and high frequency , it is efficient capacity conversion material and chip control . when the needle touch the freckle skin,it produce plasma with high temperature as 2000 °C immedicately, then it can carbide the spot freckle , removal it.

When it touch the freckle ,spot ,it will produce a small flame,we called it carbide problem skin.The micro vascular will be solidified quicly without bleeding.Then it will formed carbon scabs ,protect wound skin from infection. After about 1 week, scab will sheeding without scar.

Advantage

- Remove completely: instantly get rid of skin neoplasms and strong antiseptic
- No bleeding, with the function of blood clotting
- Wound easy to recover
- No scar left
- Ease use: it can work for about 2-3 hours after 1 time charge.
- Economical and practical: it is cheap and needle can reuse.
- Easy to take

Feature:


It produce high frequency current,biological tissue can be cutting, carbide.
The needle is soft with good flexibility.

Specification:

Input: AC110V/60HZ AC220V/50HZ
Frequency: 1.1MHz±0.3MHz

Function

Removal spot, freckle ,verruca plana, flat warts,nervus sebaceous,granuloma,
skin neoplasm,fat granule, hemangioma,eyelid

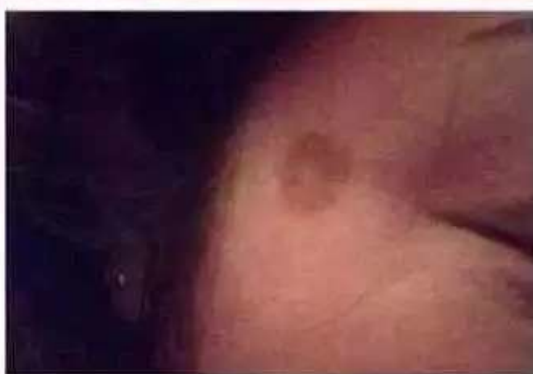



Operation

1. Clean face ,apply the anesthetic
 2. Sterlize face
 3. Needle disinfect (needle immerse in disinfectant for 5-10 minutes)
 4. Do treatment (we have video show you about how to operate)
 5. After treament, if needle have any carbide,use knife scarpe it or sand paper rub it.
- Needle disinfected before use .





Accessory list

machine	1pcs
30G needle	3pcs
charger	1pcs
manual	1pcs