

V8 Body Sculpting Beauty Machine

Model No. -- LN-1604

User Guide



Thanks for choosing our product

Please read through the manual before using

If any changes, our company reserves the right of final Interpretation

Contents

Operating Manual

Chapter 1

1.1 General Structure	3~4
-----------------------	-----

Chapter 2

2.1 Key Technology	5
2.2 Main Function	5
2.3 Preparation Before Use	5
2.4 User Interface	6 ~ 11

Chapter 3

3.1 Treatment Settings	12 ~13
3.2 Suggest Operation Method	14 ~ 16
3.3 Points For Attention	17

Chapter 4

4.1 Contraindication	18
4.2 Warranty	18
4.3 Technical parameters	18
4.4 Accesories of this device	19
4.5 Contact informations	.19

Chapter 1

1.1 General Structure

1.Vacuum Roller RF



2.40K Cavitation



3.Cryo Therapy



4.Tripolar Body RF



5.Tripolar Face RF



Chapter 2

2.1 Key Technology

- 1、 Automatic Roller Vacuum RF
- 2、 40KHz strong wave cavitation
- 3、 Cryo therapy skin firming
- 4、 Tripolar RF fat dissolve
- 5、 Tripolar RF facial treatment

2.2 Main Function

- 1.Body shaping,fat dissolving,repel the cellulite.
- 2.Lymphatic drainage,promoting metabolism&blood circulation.
- 3.Decrease the excessive water and prevent the fat from accumulation.
- 4.Tightening skin,strengthen skin elasticity,skin lift
- 5.Clear belly wrinkles,especially the stretch marks.
- 6.Complementing the collagen,skin rejuvenation.
- 7.Eliminate fine lines , eyes bags, black eyes etc.






2.3 Preparation Before Use

1. Install the attached on-hook on the machine, put the probes into the corresponding racks.
2. Check the host power cable is connected.
3. Check the key switch of machine is connected.


2.4 User Interface

Press any key to enter the main menu to enter the system.







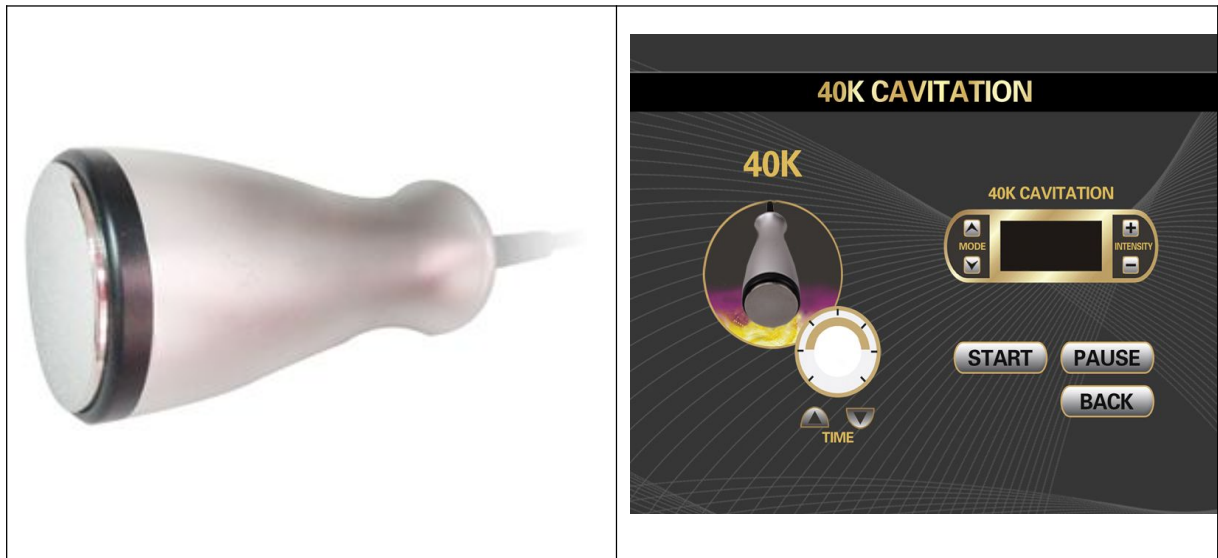

1. Click " " to enter the 40K fat exploding function.
2. Click " " to enter the RF Body treatment function.
3. Click " " to enter the RF Face treatment function
4. Click " " to enter the Vacuum Roller fat remover function
5. Click " " to enter the Cool therapy function

❖ The main menu interface for cavitation







Operation process:

1. "Time" **TIME** refers to the time of operation, which can be set by  and .
2. "Intensity" **INTENSITY** refers to the cavitation intensity, which can be set by  and .
3. "Mode" **MODE** refers to fat exploding mode selection, which can be set by  and .
4. Press **START** "to start the operation
5. Press **PAUSE** "to stop the machine
6. Press **BACK** "return back to the main interface

❖ The main menu interface for RF Body





Operation process:



1. "Time" **TIME** refers to the time of operation, which can be set by  and .
2. Adjust the RF intensity, which can be set by  and .
3. Press "**START**" to start the operation
4. Press "**PAUSE**" to stop the machine
5. Press "**BACK**" return back to the main interface

❖ The main menu interface for RF Face



Operation process:

1. "Timer" **TIME** refers to the time of operation, which can be set by  and .

2. Adjust the RF intensity, which can be set by  and .

4. Press **START** "to start the operation







5. Press **PAUSE** "to stop the machine

6. Press **BACK** "return back to the main interface

❖ The main menu interface for Vacuum





Operation process:



1. "Time" **TIME** refers to the time of operation, which can be set by  and .
2. "Intensity" **INTENSITY** refers to the Vacuum suction intensity, which can be set by  and .
3. "Mode" **MODE** refers to Vacuum suction & release mode selection, which can be set by  and .
4. Press **START** "to start the operation
5. Press **PAUSE** "to stop the machine
6. Press **BACK** "return back to the main interface

❖ The main menu interface for Cryo Therapy



Operation process:

1. "Time" **TIME** refers to the time of operation, which can be set by  and .

2. "Intensity" **INTENSITY** refers to the Cool intensity, which can be set by  and .

3. Press **START** "to start the operation

4. Press **PAUSE** "to stop the machine

5. Press **BACK** "return back to the main interface

Chapter 3

3.1 Treatment settings

➤ **Cavitation Treatment settings**

- 1.The CAVITATION treatment focuses on the abdomen, thighs (“saddle bags”) and flanks (“love handles”).
- 2.Treatments are 20 minutes long, covering a body area the size of an A4 page. If you want a larger area covered, treatment time can be extended.
- 3.Drink at least 2 litres of water before your treatment and another 2 litres in the day after your treatment (soft drink, soda, juice or tea is not water).
- 4.Do not drink ANY alcohol or caffeine in the day of treatment of 48hrs after.
- 5.Use Dry Skin Brushing daily to stimulate lymph movement (your Naturopath will show you how to do this).
- 6.Do not have a hot shower (warm is alright), spa or sauna straight after treatment.
- 7.Diet: Follow a fat free, low starch, low sugar diet for best results (your Naturopath will give you an info sheet).
- 8.Exercise: Do brisk exercise for at least 20 minutes after your treatment to stimulate lymph movement.
- 9.An average of 2 cm reduction in circumference was achieved after a single treatment of the abdomen, outer thighs, or flanks.

➤ **RF Treatment settings**

- 1.RF treatment takes 20 minutes in generally. Treatments can be applied to the eyes, arms, buttocks or thighs to aim for skin tightening, reduced improvement in stretch marks and fat dissolve.

2.The optimal frequency for using Focus Fractional RF is 2 -3 times a week for 6-8 weeks. It is recommended that the Focus Fractional RF device not be used more than once every other day.

3.Letting the skin rest in between each treatment is important to allow the natural process of the skin's collagen renewal to take full effect.

4.Keeping in mind that the treatment does not stop the aging process:

- For Cellulite - results typically last a few months if a regime of diet and exercises is followed.

- For longer term results the customer needs to return for maintenance treatments every 4-8 weeks.

- For Skin tightening - results typically last for a few months, for longer term results the customer needs to return for maintenance treatments every 4-8 weeks.

➤ **Vacuum Roller Treatment settings**

1. Usually 5 to 10 minutes for each part(depending on thickness of fat and health status.)

2. About Vacuum Suction treatment:

- Vacuum suction treatment is applied to the surface of the skin tissue causing lift and stimulation of the under-lying structures.

- Vacuum suction is aimed at improving blood flow and Lymph circulation.

- Lymph fluid transports metabolic waste and fatty materials via the lymph nodes and lymphatic system.

Treatment Tips:


We recommend do the vacuum suction roller massage first , and the do the Cavitation cellulite reduction treatment . Finally , use RF tighten the skin . Related Gel is neede in every treatment.

3.2 Suggest Operation Method


✧ THIGH

Treated time: 30 minutes


1. From down to upper, pushing to the groin to dredge the lymph.

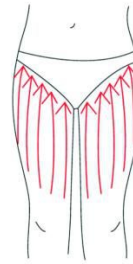


2. From down to upper, circling push by anticlockwise, can help decomposing fatness.

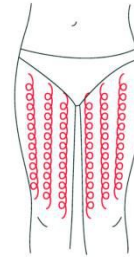

3. Also can push by come-and-go to decompose fatness.

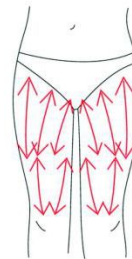


4. Pull from the knee and muscle texture to upper, can improve the curve.

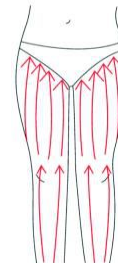


✧ BUTTOCKS

Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.

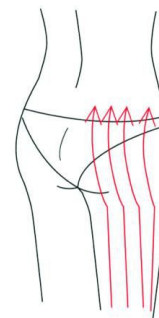


2. From upper to down, by anticlockwise gesture, pull come-and-go.
3. Along the muscle of arm, pull up to lift and tight the muscle.

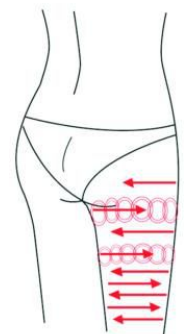


✧ **BACK**

Treated time: 20 to 30 minutes


1. First by come-and-go, pushing the bladder nerve 2 to 3 times.

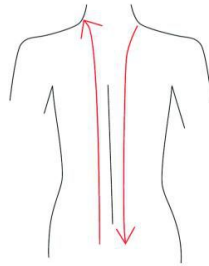


2. Circling by anticlockwise to stimulate the underarm lymph node.

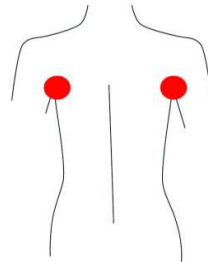

3. On the back, by anticlockwise circling, can help to decompose the fatness.

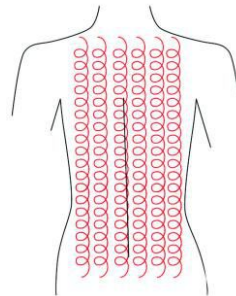

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back curve.

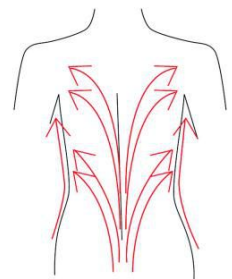

✧ **ARM**

Treated time: 30 minutes

- BACK SIDE

1. From the inner elbow to the armpit, doing lymph drainage.



2. Circling to stimulate the lymph node.

3. By anticlockwise or come-and-go pushing, decomposing the fatness

4. From the elbow to the oxters, doing tightening gesture.



- Facade


1. From the elbow to the oxters, doing tightening gesture.




✧ Abdomen

Treated time: 20 to 30 minutes


1. From small to large, by clockwise direction, circling around the navel can help peristalsis of the large intestine.

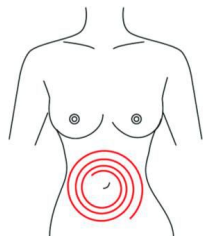


2. From small to large, by anticlockwise direction, circling push down.

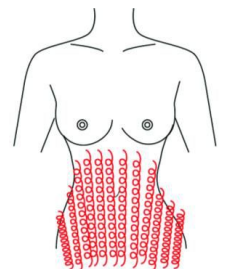

3. From the belly and muscle texture, pull to the groin.

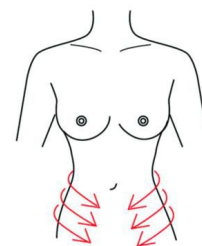

4. Through lymph drainage gesture, taking the toxin to the place of groin.

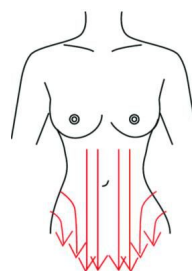

3.3 Points for attention

1. The machine can be operated by professionals or personals.
2. The patient cannot receive the treatment if he/she has following medical electronic equipment:
 - Implanted cardiac pacemakers or other implanted electronic devices;
 - Life-support artificial heart-lung machine or other equipment;
 - Portable ECG measurement device;
3. The following patients shall have the professional physician's consent before using the machine
 - Acute disease patient;
 - Cancer patient;
 - Infectious disease patient;
 - Pregnant women;
 - Heart disease patient;
 - Fever patients;
 - Patients receiving another treatment or those with physical anomalies;
4. Careful attention should be paid to following patients when using this machine:
 - Infants and young children;
 - People with sensory or physical impairment;

Chapter 4

4.1 Contraindication

Please avoid the treatment from these people :

1. Pregnant women or women who may be pregnant.
2. Scarring, infection of the treatment area
3. History of keloid scarring
4. Use of medication known to induce photosensitivity
5. Fragile skin
6. Patients undergoing the surgical procedure without consulting with a surgeon
7. Patients with tumors
8. Patients with serious disease
9. Patients with low/high blood pressure

4.2 Warranty

1. Manufacturer warrants defects in materials and workmanship for a period of 1 year from the date of original purchase.
2. Manufacturer won't bear any responsibility:
 - A. Damage caused by accident, abuse, misuse or misapplication, it will be repaired with repair fee.
 - B. Damage caused by service performed by anyone who is not technician or qualified by manufacturer.
 - C. A product or part that has been modified without any permission of manufacturer.

4.3 Technical Parameters

1. Host display: 8 "TFT color touch screen
2. Voltage : 220V/50Hz or 110V/50Hz (according to the mark on the machine)
3. Power: 300w
4. Vacuum: 650mmHg
8. Cryo temperature: 0°C to 10°C
9. Fat exploding frequency: 40KHz
10. RF frequency: 1MHz(Face) , 5MHz(Body)

4.4 Accessories of this device

1.Power supply line	1pcs
2.Handle support	1pc
3.User manual	1pcs
4.Cooling handle	1pcs
5.Cavitation handle	1pcs
6.RF handle	2pcs
7.Vacuum handle	1 pc
8.Emergency switch	1pc