

BEAUTY EQUIPMENT

Bipolar RF Ultrasonic Liposuction Cavitation

Vacuum Slimming Machine



Theory

With collective strong sound wave head, strong sound wave of 40000HZ may be emitted to vibrate fat cells at top speed and produce numerous vacuum air pockets inside and outside the fat cells, robustly impact fat cells to generate introverted blast and disintegrate triglyceride into glycerol and free fatty acids. Then RF waves at frequency of 1M HZ is used for exhausting the integrated glycerol and free fatty acids through hepatoenteral circulation. Finally, vacuum RF and energy electrode are used for positioning and tightening fat. In physics, it is known as "cavitation". Micropore introverted blast inside and outside cell may lead to enhanced molecular motion and a higher energy level and this will finally cause fat cell rupture and thereby achieve the effects of body building and losing weight.

1. Dissolve fat, lymphatic drainage, firm skin and enhance skin elasticity

Negative pressure RF liposuction head

Being integrated with the most advanced RF technology and radio frequency, the instrument may directly reach the deep-seated fat body and have the excellence of targeted positioning RF. In the fast-active state, fat cell tissue may generate friction heat, increase local temperature and remove excess fat and toxin from the body through sweat gland, enterohepatic circulation and lymph and ultimately achieve the effect of dissolving fat. With controlled depth technology, inlaid diamond particles of different sizes may rub skin along its pattern, adjust suction strength by means of vacuum draw and rub with negative pressure strong force to directly explode thick fat. Thus, the effects are very obvious: Enhance blood circulation, accelerate the decomposition of fat, take away wastes, enhance the effect of removing cellulite, effectively improve the state of areolar tissue, remove stubborn fat and improve the sufficient absorption condition of diet products.

2. Strong force explosion fat and fat removal

Strong sound wave explosion fat head With collective strong sound wave head, strong sound wave of 40000HZ may be emitted to human body for impacting fat cells fiercely and causing friction motion between fat cells. This may cause effective consumption of calories and moisture in fat cells and reduce the size of fat cells. What's more, sound wave vibration may cause fierce impact of fat cells to make them be exploded instantaneously, reduce the amount of fat cells and thereby achieve the effects of removing fat.

3. Body Vacuum RF Probe

For body and face re-shaping. Weight loss Vacuum+RF fat-explosion machine can promote tissue metabolism, repel the cellulite, which make fat granule in different depths and shocking make fusion energy, and the deepest receptor can get granule in 3CM thick of skin, and rapid expansion and the rapid heating of the cell wall that exceed the elastic limits and arises cells broken, so that the combustion of fatty acids at the same time broken out of a pot, not only the burning consumption of fatty acids but also more directly emptying so rapidly reduce the size of granule

4. Facial wrinkle elimination, tightening and lifting

Facial RF head A capacitive coupling electrode is used for transmitting radio-wave energy and generating an electric field for entering into subcutaneous tissue through the skin surface. In the high-frequency of 450khz, this electric field changes its polarity for 450,000 times per second. To reply to the rapid change in electrode, directions of electrical particles the skin are also changed. By this time, natural electric resistance in the subcutaneous tissue moves and generates heat energy. As the papillary dermis collagen may immediately shrink when the temperature is within the range from 60 to 70 Celcius degree, after wrinkle treatment, client may immediately sense the skin tightening effects as it is being lifted and firmed. When collagen is produced continuously, thickness and density of the skin papillary dermis may be increased to remove wrinkles, eliminate scars, restore skin elasticity and gloss and make it be blonde and smooth. While collagen is increased, fresh skin is generated in the position of operation and wrinkles are removed by large amount of cells. In addition, when cortex without elasticity or that with thick horny layer in the area with wrinkles is separated, surrounding skin will also be renewed.

Features & Functions

Features

1. The entire process is completed without the need of surgery and anesthesia.
2. Does not cause the phenomenon of uneven skin.
3. Does not cause the phenomenon of bleeding, swelling and blood stasis.
4. Without side effects and the risk of gaining the weight back, the results are rather obvious.
5. Non-invasive treatment does not affect the normal work and life.

Functions

Firm the arms, legs, thighs, buttocks, waist, back, abdominal muscles and reshape the body figure

*Improve and treat the orange peel like fatty tissues of different serious levels

With the effects of soothing, decompression and pain relief (such as joint pain, sciatica), promote the regular mechanical aerobic exercise of skin tissue, rapidly relieve stress and remove fatigue.

Firm, lift and compact the skin and mould the face shape

Circulate and activate cells, promote metabolism and so that 100% more collagen and elastic fibers may be gained for improving the facial and body skin

Package Including

Multipolar RF head(Tripolar RF) X 1

40KHz Cavitation Head X 1

Vacuum Bipolar Head x 1

Bipolar RF Head for face X 2

Power cord x 1

Operation procedures





Vacuum RF liposuction head

Adjust the SUCK Energy level , and the PUT Energy level, (normally Suck energy should more than the Put energy)

Adjust the RF level from 0 to 1, or 2....(you can try on our palm)

Notes:

1. Wipe the slimming essential oil on the skin and do massage before start the treatment.
2. Do not use the slimming gel for the smoothly, because the gel is easy block the vacuum head, it is better to use the slimming essential oil.
3. Do not put so much slimming oil on the skin by each time..pay attention to prevent the oil sucked into the vacuum head, or it will damage the head
4. Adjust the intensity from low level to high level. Or try the head on hand in advance.
5. Turn off the instrument probe when it is not in use.

40kHz Ultrasonic Head:

Press "TIME SET" to adjust the time, select the strength of "LOW" , "MIDDLE" or "HIGH" by clicking "-" to decrease the strength or click "+" to increase it. Press "PAUSE" to start the strong

Mode a.b.c.d means that:

"A" is continuous working for 3 sec and discontinuous for 2 sec.

"B" is continuous working for 4 sec and discontinuous for 3sec.

"C" is continuous working for 1 sec and discontinuous for 1 sec.

"D" is continuous working for 0.5 sec and discontinuous for 0.5sec.

Body RF head

Panel: Press "RF CHANNEL" in the display screen to select body RF output, select the suitable intensity (adjust according to the customer's acceptable level, 3-6 is ok.), and at last press PAUSE key. The face RF starts to work.

Tips: use some slimming essential oil on the skin for massage, then use the RF head to burn the fat..move the head on the skin and do the massage..

Facial RF head

Press "TIME SET" to adjust the time,

press "RF ENERGY" and then "+" to increase the RF intensity or "-" to decrease RF intensity. Adjust to a proper level (according to acceptability of clients. General y speaking, levels 3 - 6 are OK).Spread gel or wrinkle smoothing essence on the treated positions for hairdressing (general y 20-30G each time), operate with the facial RF head for 15-20 minutes with mainly the spiral movement on face.

Operation Guides

THIGH

Treated time: 30 minutes

- 1.From down to upper, pushing to the groin to dredge the lymph.
- 2.From down to upper, circling push by anticlockwise, can help decomposing fatness.
- 3.Also can push by come-and-go to decompose fatness.
- 4.Pull from the knee and muscle texture to upper, can improve the curve.

Treated time:20 to 30 minutes

- 1.Along the lymph direction, pulling to the waist.
- 2.From upper to down, by anticlockwise gesture, pull come-and-go.
- 3.Along the muscle of arm, pull up to lift and tight the muscle.

BACK

Treated time: 20 to 30 minutes

- 1.First by come-and-go, pushing the bladder nerve 2 to 3 times.
- 2.Circling by anticlockwise to stimulate the underarm lymph node.
- 3.On the back, by anticlockwise circling, can help to decompose the fatness.
- 4.Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back curve.

ARM

Treated time:30 minutes

BACK SIDE

- 1.From the inner elbow to the armpit, doing lymph drainage.
- 2.Circling to stimulate the lymph node.
- 3.By anticlockwise or come-and-go pushing, decomposing the fatness.
- 4.From the elbow to the outer, doing tightening gesture.

Face

1. From the elbow to the outer, doing tightening gesture.

Abdomen

Treated time:20 to 30 minutes

- 1.From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.
- 2.From small to large, by anticlockwise direction, circling push down.
- 3.From the belly and muscle texture, pull to the groin.
- 4.Through lymph drainage gesture, taking the toxin to the place of groin.

Notice:

Those do not use the machine:

- 1.Pregnant women or women during menses.
- 2.Epileptic

3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Whom with heart diseases or with heart pacemaker..
7. Whom with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
10. Whose body always take much inner hot.
11. Who has the genetic hypersensitivity.

Attention:

1. Be sure to use the Slimming essential oil
2. Avoid knocking the head.
3. For avoiding burning the head, during operation, please prepare enough ultrasoinc gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected wel .
7. If you don't use the machine, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. Please do not use the machine over 1 hour, after one hour, please turn off the machine and cooling it .
10. During the operation, be sure not to accept other treatment.

Tips:

*Important: It is necessary to drink about 2 liters of water per day, 2 or 3 days before the first session ,and after the treatment, should drink more water for detoxing... it helps for slimming.

*The lipocavitation-cons is indicated in the following cases:

Pregnant women, people with a pacemaker, defibrillator or metallic materials,
On the scars of less than 3 months
For people with diabetes and / or cardiovascular disease,
Among those who had previous thrombosis or phlebitis
For persons on anticoagulant therapy,
In patients with tumors or followed by a surgical procedure

*During any function treatment , the guests and beautician must take off all the metal jewelry, so that can avoid appear the phenomenon of current increase.

Technical Parameters

Power supply Input:110V/220V 60HZ/50HZ

Power :130W

40K cavitation head:

Power supply output:150V

Frequency:40KHz

Power:25W

Vacuum+Bipolar RF head:

Power supply output:66V

Frequency:450K

Power:95W

Body RF head:

Power supply output:66V

Frequency:550k

Power:50W

Tripolar RF head:

Power supply output:66V

Frequency:450K

Power:35W

Face RF head:

Power supply output:66V

Frequency:450K

Power:35W

SERVICE AND Warranty

Service

We will guarantee the quality of our equipment, normally there is no quality problem.

1. If there is a deficiency, please email or call us on the reasons, it would be best if this can be dealt with by email or telephone. Because some of the problems are caused by using the machine in a wrong way.

2. We suggest the client to take photos of the area where the problem occurs, we will pass them to our technician, our technician will give a satisfactory answer within 1-7 working days

Warranty

from the date of purchasing according to the receipt.

Warranty for machine body:

1 year warranty for the machine body (not including the consumables handpieces), if there have any malfunction with the machine body, factory provide the circuit board or any other accessories freely, customer bare shipping charge.

Warranty for the Handpieces:

3 months warranty for the handpieces, if there have any malfunction with the handpieces, factory provide the handpieces freely, customer bare the shipping charge.

Any other non-warranty parts or out of warranty period, can also be responsible for the maintenance and updating, such as Spare parts and consumables cost, if need purchase, please contact your dealer.

2, the following users personal reasons caused by the fault, do not provide free warranty service:

- a. Malfunctions caused by abused operation
 - b. Malfunctions caused by modification or repair by anyone or not authorized by our company; or the original serial number tag or product identification markings have been altered or removed
 - c. Malfunctions caused by other artificial factors like move or tear down in proper motion, or collision, etc.
 - d. Malfunction caused by uncontrollable natural force
 - e. Energy reduce caused by wet environment and dust
 - f. Consumable parts of the equipment: handpieces
- 3, the company's product design specifications are subject to change without notice, if in doubt, please contact the company

Packing Size: 44*43*44CM

G.W:18Kg