

# LN-1631 5M RF machine

## User manual



<u>Warning</u>	<u>1</u>
<u>Parts introduction</u>	<u>2</u>
<u>Annexes</u>	<u>2</u>
<u>Operation</u>	<u>3</u>
<u>“RF” Function</u>	<u>3</u>
<u>Forbidden group</u>	<u>4</u>
<u>Technique parameters</u>	<u>4</u>
<u>Annex List</u>	<u>4</u>

---

---


## **Warning**

Dear user:

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:

- ◆ ◆ Please read this instruction in detail, and keep it well for further reading and consulting.
- ◆ ◆ Please follow the instruction to fit and operate the machine correctly.
- ◆ ◆ Please do not remove and change any accessories of the machine.
- ◆ ◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆ ◆ It forbids others to change and maintain the machine except the professional people.
- ◆ ◆ Please contact us if there is any trouble during operation.

## Parts introduction

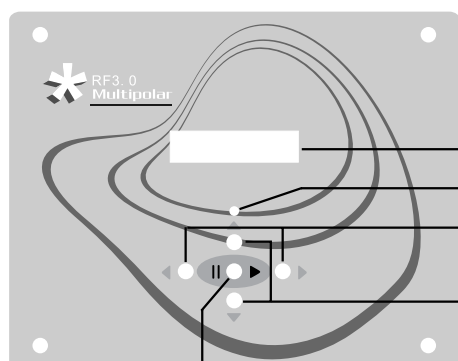

- 1.LCD screen
- 2.function selection
- 3.adjusting button
- 4.start/pause
5. indicator light
6. power line outlet
7. general power switch
8. RF slimming head jack for body
9. RF slimming head jack for eyes

## Annexes



Body RF slimming head



Eyes RF slimming head

Any part of the instrument made of the PMMA material (for example: accessories shelf, panel, transparent handles, etc.), Do not use alcohol or wipe clean with a corrosive solvent.

---

## Operation

- 1.Connect the accessories to the host.  
Insert Body RF slimming head to [8] jack.  
Insert Eyes RF slimming head to [9].  
Insert power line to [7].
- 2.Turn on general power switch[6],the LCD screen will show the time:30:00,channel1
3. When start or pause the machine, click"function selection" to change different functions.  
The selected will light up and then press the [3] to adjust the time and channel.  
(1)after setting the working time. the minutes window will light up and this machine works during 1~60minutes.  
(2)after setting the channel, the channel window will light up and this machine will work in two channel: channel 1 or channel 2.  
(3)after adjusting the time and channel, press the "functions selection" key once again and the LCD screen will not light up.
- 4.Click the "start/pause"[4] to start work and the green lamp will light up at the moment.
- 5.during the working time, please press the key [3] to adjust the working time and working intensify.  
(1)please insert the intensify range in the channel 1:intensify 1(weak)-5(strength).  
(2) please insert the intensify range in the channel 2:intensify 1(weak)-9(strength).  
(3)after adjusting the channel, press the adjusting key once again to make the lamp stop twinkling
- 6.when the predetermined time is up, the machine will stop working automatically. At the same time, the indicator light will stop twinkling and the buzzer will ring six times to remind the guests of the time. at this moment it can come back to the initial state with pressing any key.

## "RF" Function

### "RF"Function procedure

1. after turning on the machine, please setting the working time(20 minutes for one part) and channel 1 or channel 2.
2. daub the gelatin on the part of body where need care and click the start/pause key to turn on the machine. At this time, press the key [3] to adjust the output intensify:  
(1)channel 1:intensify 1(weak)-5(strength)  
(2)channel 2:intensify 1(weak)-9(strength)
3. make the electrode probe close to the skin for slow-moving, do not stop.
- 4.the machine will stop working when the working time is up.
5. Clean the leftovers with hot towel, use warm towel to clean the RF slimming head.

### caution

- 1.When start "RF" function, make sure that the lines not tangled.
- 2.Enough gel is needed to avoid pains.
- 3.Do not wear any aglet, both for the beautician and the users. do not use on parts with metal in filling inside.
- 4.Do not make the RF electrode stay on any part without moving

---

## Forbidden group

1. People with heart disease or high blood pressure or configured cardiac pacemaker.
2. Patients with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic disease, trauma or who is bleeding.
4. Pregnant women.
5. Medical Plastic parts, or parts with artificial infilling inside.
6. People with metal inside the body(include metallic tooth).
7. Patient with skin inflammation or with edema.
8. People with an abnormal immune system.
9. Numb or insensitive to heat.

## Technique parameters

Voltage: AC220/50Hz

Power:<100W

## Annex List

1. Host	1PC
2. Power line	1PC
3. User manual	1PC
4. Eyes RF slimming head	1PC
5. Body RF slimming head	1PC
6. hexagonal screwdriver	1PC
7. hexagonal screw	4PC
8. accessory frame	1PC