

M6

# MANUAL



# Catalog

Parts Introduction.....	1
Annex.....	1
Parts specialties narration.....	2
General introduction.....	3
Operation.....	3
Interface explain.....	4
Vacuum&roller treatment.....	5
Caution.....	6
Technique parameters.....	7
Accessories list.....	7
Assemble step.....	8

Parts Introduction



- 1.LCD screen
- 2.General power switch
- 3.Power supply line jack

Annex



Vacuum big roller



slip roller




Pattern roller



Vacuum small roller



# Parts specialties narration

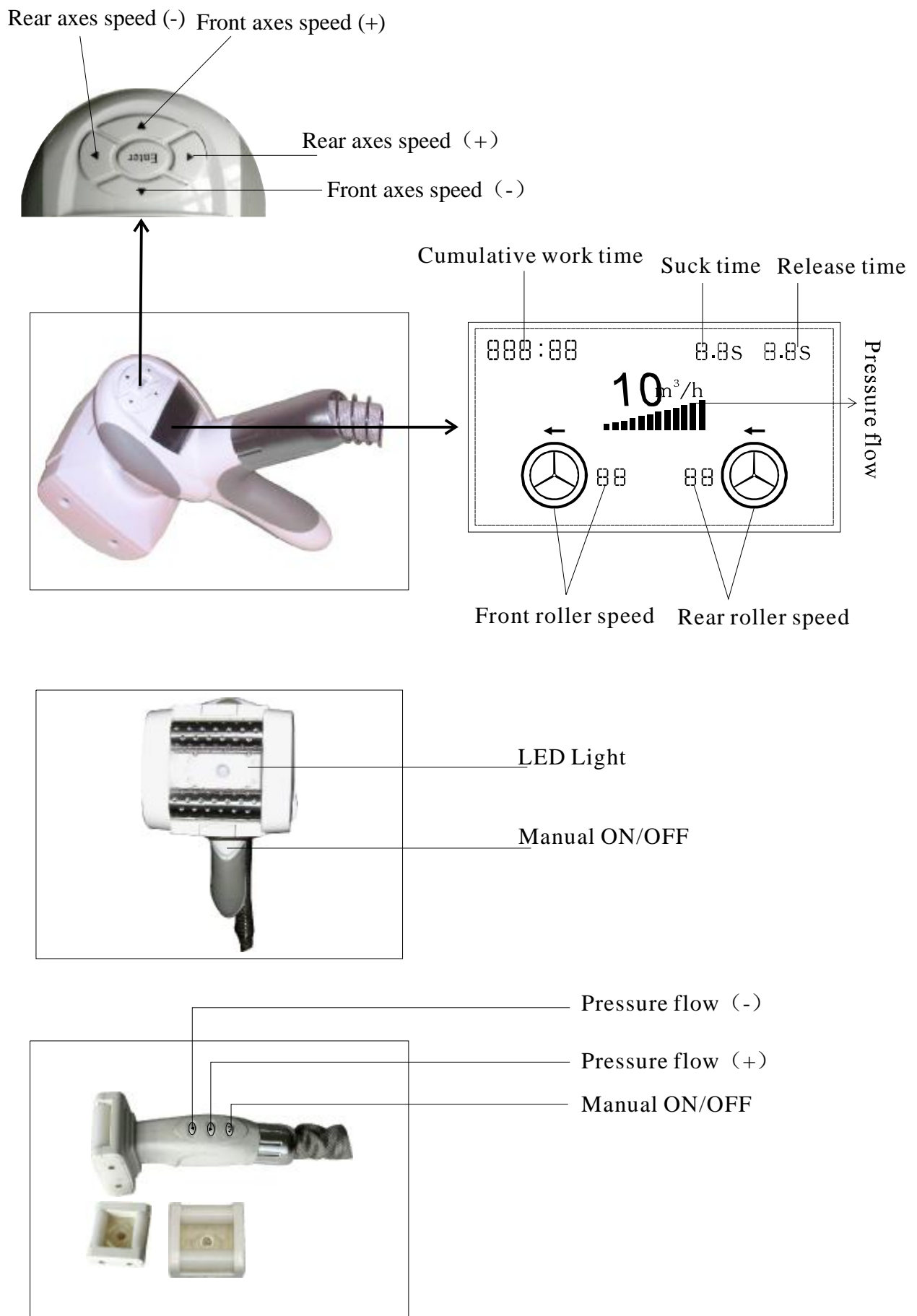

## General introduction

M6 liposuction ultrasonic fat removal instrument has the functions of vacuum suction, motorized intelligent roller and LED light therapy, it is a high-tech micro-computer control machine that has a excellent efficacy in slimming, body shaping and skin beauty.

The machine use vacuum suction to lift up skin together with subcutaneous fat and muscle tissue. And to do deep massage as sucking, pinching, pushing, pulling, rolling on body using the roller, which make skin and adipose tissue to do aerobic exercise, to redouble enhance blood circulation and lymphatic detoxification function. And the back and forth deep massage can speed up fat burning, and transfer fat into fatty acids and then discharge out through lymphatic system. It can decompose subcutaneous fat, consume accumulative fat, to prevent and reduce the formation of cellular organization effectively, thus to achieve effect of body shaping and slimming. At the same time, it strengthen capacity of skin collagen and elastic fiber regeneration, make skin healthier, more luster and flexible.

## Operation

- 1.Place accessories well and connect to main body.
- 2.Connect power supply line, turn on general power switch.
- 3.Click the screen System entering standby mode, the screen displays interface as Figure 1.
- 5.Function select: Fat rolling(vacuum suction& big roller)  
Fat slip(vacuum suction& small roller)
- 6.Set work time(1:00 ~ 120:00 min), click "t " "u" key in interface as Figure 4 to regulate.
- 7.Click "Start / Pause" to start working.
- 8.To choose work mode, regulate vacuum flow, roller speed and output intensity.
- 9.The machine will suspend automatically if the setting work time is over and enter interface as Fig 4.  
To continue to use, click on the screen, to enter interface as Figure 5, re-set work time, and then select function.

## Interface explain

Operation:

1. Click different functions on screen.
2. Move the "☞" cursor to place need regulate.
3. Click "t" "u" to regulate.

### fat rolling interface

Vacuum flow:  $1\text{m}^3/\text{h} \sim 10\text{m}^3/\text{h}$

Vacuum mode: P0~P9 (P0—continuous suction  
P1~P9—pulse suction)

Suck time: 0.1S~8.0S

Release time: 0.0S~5.0S

Fat rolling mode: +/-B1 pull back towards  
-/+B2 push forwards  
+/-B3 roll inward


Click "set"

Roll-forward speed: 1~10


Roll-backward speed: 1~10

Photon mode: Combine light, blue light,  
red light, close

### fat rolling interface

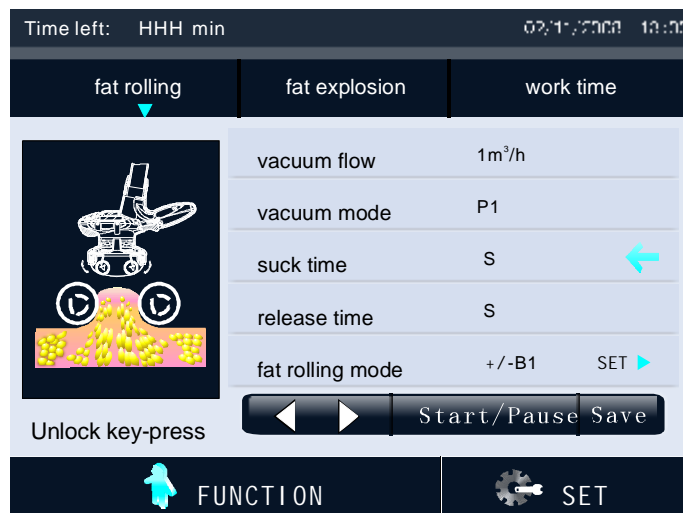

(Fig 1)

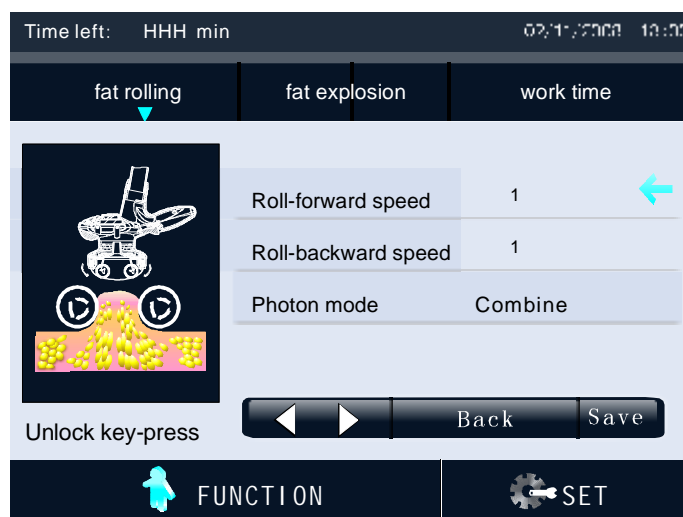

(Fig 2)

### fat slip interface

Vacuum Flow:  $2\text{m}^3/\text{h} \sim 8\text{m}^3/\text{h}$


Vacuum Mode: P0~P9 (P0—continuous Suction  
P1~P9—Pulse Suction)

Suck Time: 0.1s~8.0s


Release Time: 0.0s~5.0s

▲ When release time regulated to be "0", the suck time and release time on the screen shows— ☐ — S. It means the vacuum mode is continuous suction at present.

### fat slip interface

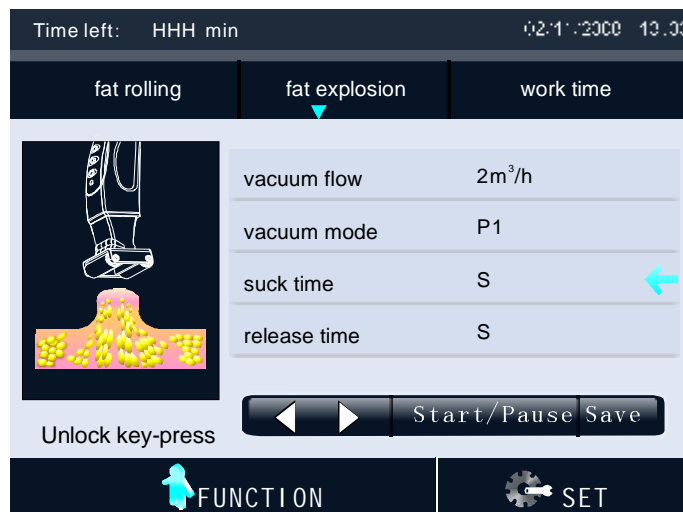

(Fig 3)

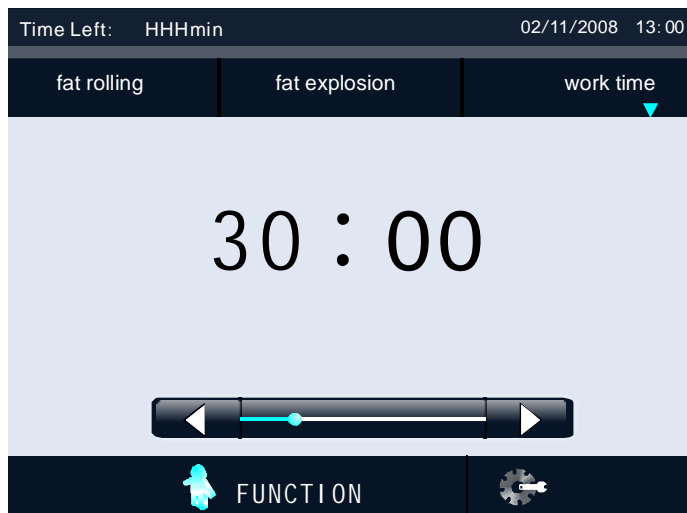

(Fig 4)



(Fig 5)

### Vacuum&roller treatment

- 1.Customer put on special slimming clothing after taking a shower.
- 2.Operation-select fat rolling function, cling the big roller to the slimming clothing.  
 Vacuum mode: P1~P9—pulse suction  
                   P0—continuous suction  
 Fat rolling mode: -B1(pull back towards)+B2(push forward)+B3(roll inward)  
 Fat rolling speed: 1(slow)~10(fast)  
 Suggestion: Decompose fat, burn fat-B3+pulse suction+ red light  
                   Lymph de-toxin, edema elimination-B2+pulse suction+ light combination  
                   Slimming and body shaping, skin tightening- B2+continuous suction+ red light  
                   Cellulite elimination, massage relax-B1(B2)+pulse suction+ blue light
- 3.Choose the fat slip function, use vacuum& small roller, fit for breast, arms, crus, thigh, cervix.

## Caution

1. Do not eat two hours before treatment.
2. The customer needs to put on special slimming clothing, do not use any products.
3. To regulate vacuum flow according to the guests' tolerance, from weak gradually to strong.
4. Vacuum & big roller suit for thick muscle, fleshy part, or substantial subcutaneous tissue. The thin, uneven parts not fit for use. Do not on head, face, heart and other parts of the operation.
5. As people have different skin and health status, there may be red mark after operation, this is a normal phenomenon, may subside on its own.
6. Customers need to drink eight glasses of water every day (about 250ml/cup) to help drain toxins away.
7. The filter must be replaced after using for 40 hours (cumulative), or it will affect the suction.

Period of treatment: 20 times/a period of treatment    3 times/week    30~60min/times

Change the filter: open the lid as the arrowhead indicates, take the filter out, and change a new one, fasten the lid in the opposite direction of the arrowhead.

Notes: filters of big (small) roller should be changed every 40 hours





## Technique parameters

Voltage:110-230V/50/60Hz

Host Power $\leq$ 1000W

Vacuum Moment:5000g.cm

Vacuum $\leq$ 10h/m<sup>3</sup>

Fiber<5Mw

## Accessories list

1.Host base	1set
2.Host upside	1pc
3.Power supply line	1pc
4.Use manual	1pc
5.Host upside connect screw (M4*16)	2pcs
6.Upside accessories screw (M6*12)	2pcs
7.Hexagon screwdriver (M3,M5 each 1pc)	2pcs
8.Filter	2pcs
9.Slimming clothing	1set
10.Airproof circle (8*1.9)	4pcs
11.Airproof circle (19*2)	4pcs

