

M8

# MANUAL




# Catalog

Parts Introduction.....	1
Annex.....	1
Parts specialties narration.....	2
General introduction.....	3
Operation.....	3
Interface explain.....	4
Fat eliminate treatment.....	6
Vacuum&roller treatment.....	6
Caution.....	7
Technique parameters.....	8
Accessories list.....	8
Assemble step.....	9

Parts Introduction



- 1.LCD screen
- 2.IC card jack
- 3.M80 slimming head jack

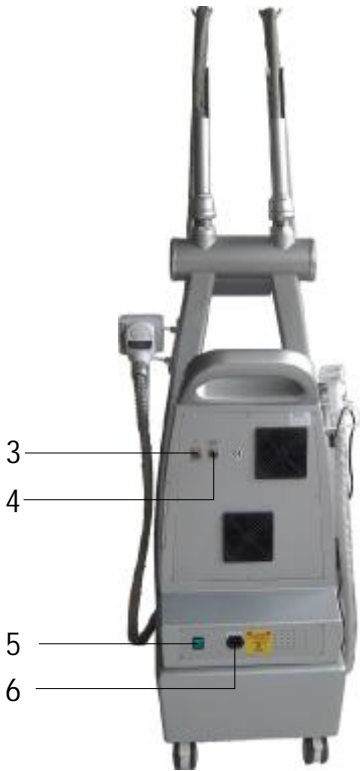


- 4.Service brake jack
- 5.General power switch
- 6.Power supply line jack

Annex

		
Vacuum big roller	slip roller	Pattern roller
		
Vacuum small roller	M80 ultrasound slimming head	Service brake

# Parts specialties narration

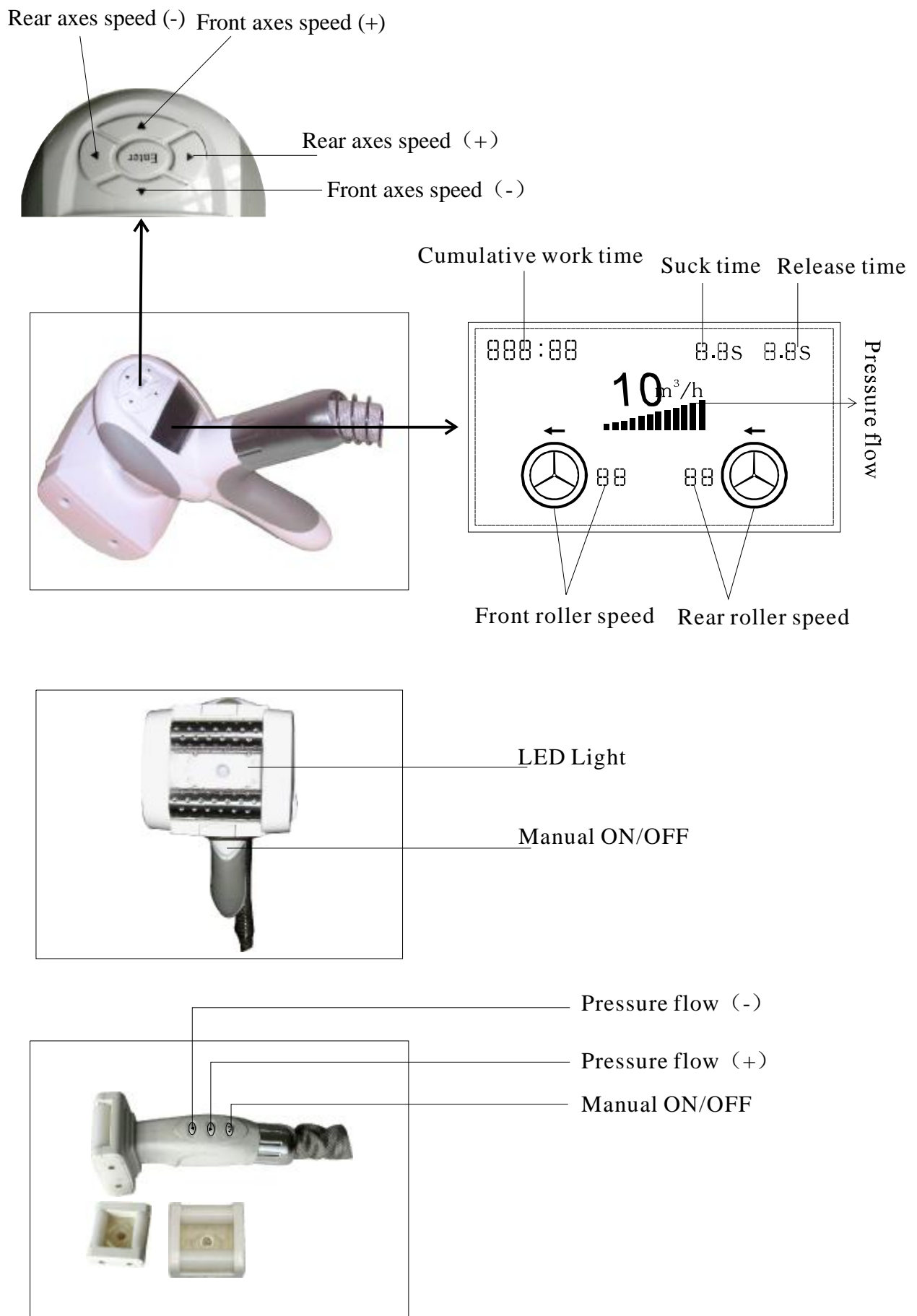

## General introduction

M8+2 liposuction cavitation machine has the functions of vacuum suction, motorized intelligent roller, 40KHz strong ultrasound and LED light therapy, it is a high-tech micro-computer control machine that has a excellent efficacy in slimming, body shaping and skin beauty.

The machine use vacuum suction to lift up skin together with subcutaneous fat and muscle tissue. And to do deep massage as sucking, pinching, pushing, pulling, rolling on body using the roller, which make skin and adipose tissue to do aerobic exercise, to redouble enhance blood circulation and lymphatic detoxification function. And the back and forth deep massage can speed up fat burning, and transfer fat into fatty acids and then discharge out through lymphatic system. It can decompose subcutaneous fat, consume accumulative fat, to prevent and reduce the formation of cellular organization effectively, thus to achieve effect of body shaping and slimming. At the same time, it strengthen capacity of skin collagen and elastic fiber regeneration, make skin healthier, more luster and flexible.

Use of 40KHz low-frequency sound waves emit strong vibration causing a strong impact on by lipocyte. Effectively deal with thick stubborn fat accumulation. To form cavitations in fat cells both inside and outside, which generated positive and negative changes in the fat cell membrane alternately be uneven both within and outside the natural burst, to reduce the size of fat cells and reduce the number of fat cells. And the implosion caused by micro gaps inside and outside cells will promote molecular motion, makes a high energy level, which finally led to break of fat cells.

## Operation

- 1.Place accessories well and connect to main body.
- 2.Connect power supply line, turn on general power switch.
- 3.Click the screen, insert effectual IC card correctly.
4. System initialization, wait for about 10 seconds then entering standby mode, the screen displays interface as Figure 1.
- 5.Function select: Fat rolling(vacuum suction& big roller)  
Fat slip(vacuum suction& small roller)  
Fat explosion(M80 ultrasound head)
- 6.Set work time(1:00 ~ 120:00 min), click "t " "u" key in interface as Figure 5 to regulate.
- 7.Click "Start / Pause" to start working.
- 8.To choose work mode, regulate vacuum flow, roller speed and output intensity.
- 9.The machine will suspend automatically if the setting work time is over and enter interface as Fig 6.  
To continue to use, click on the screen, to enter interface as Figure 5, re-set work time, and then select function.