

## Interface explain

Operation:

1. Click different functions on screen.
2. Move the "☞" cursor to place need regulate.
3. Click "t" "u" to regulate.

### fat rolling interface

Vacuum flow: 1m<sup>3</sup>/h~10m<sup>3</sup>/h

Vacuum mode: P0~P9—(P0 continuous suction  
P1~P9—pulse suction)

Suck time: 0.1S~8.0S

Release time: 0.0S~5.0S

Fat rolling mode: +/-B1 pull back towards  
-/+B2 push forwards  
+/-B3 roll inward


Click "set"

Roll-forward speed: 1~10


Roll-backward speed: 1~10

Photon mode: Combine light, blue light,  
red light, close

### fat rolling interface

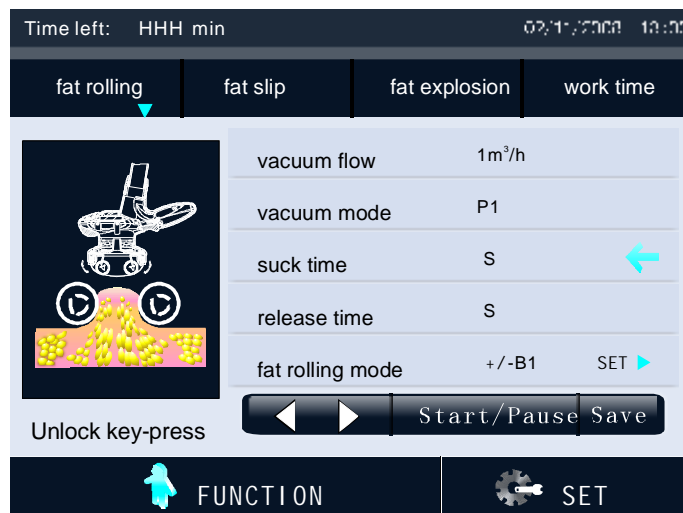

(Fig 1)

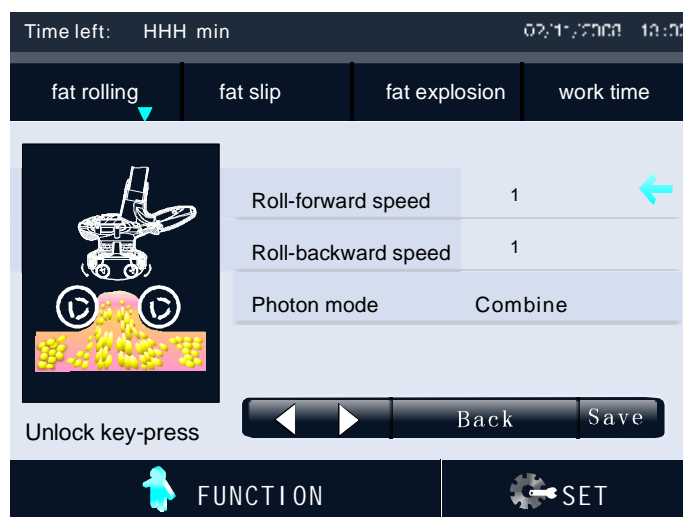

(Fig 2)

### fat slip interface

Vacuum Flow: 2m<sup>3</sup>/h~8m<sup>3</sup>/h


Vacuum Mode: P0~p9—(p0 continuous Suction  
P1~p9—pulse Suction)

Suck Time: 0.1s~8.0s

Release Time: 0.0s~5.0s

▲ When release time regulated to be "0", the suck time and release time on the screen shows— ☐ — S. It means the vacuum mode is continuous suction at present.


### fat slip interface

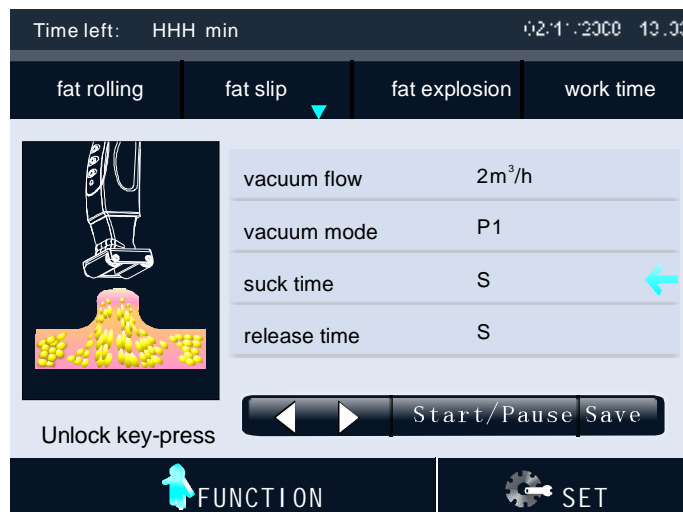


(Fig 3)

fat explosion interface

Output intensity: 5w/cm<sup>2</sup>~25w/cm<sup>2</sup>

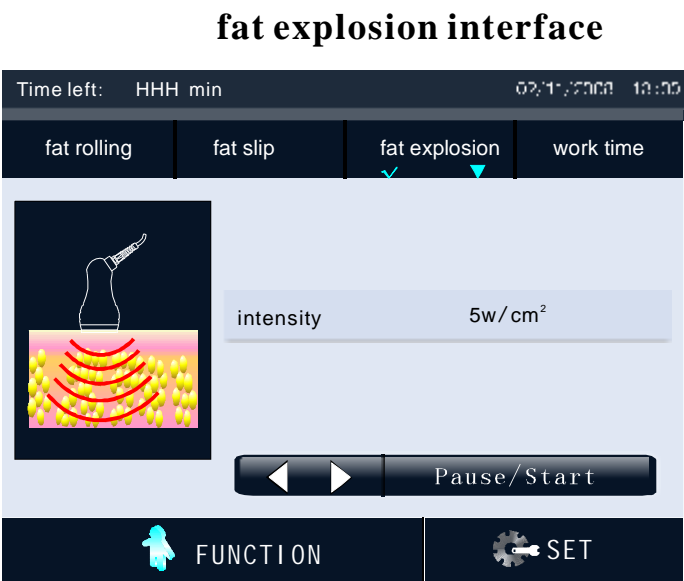

(Fig 4)

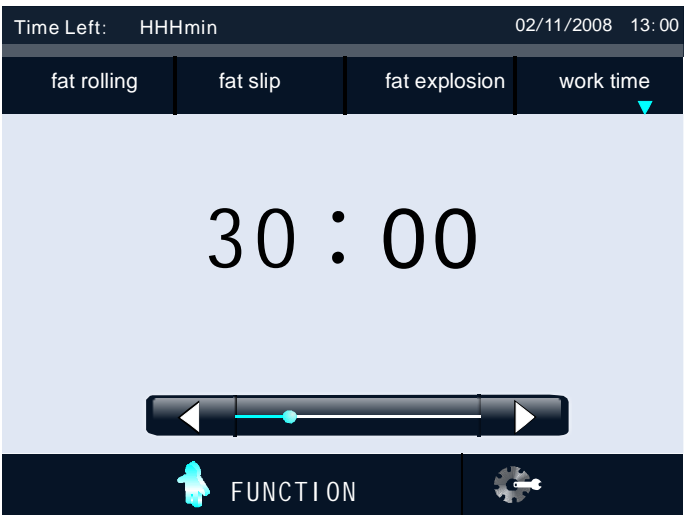

(Fig 5)



(Fig 6)

## Fat eliminate treatment

1. Apply gel on fleshy parts(eg.abdomen,hip,thigh) The amount depends on moving flexibility of the ultrasound head.
2. Set work time(10 min every part), regulate output intensity ( $5\text{w}/\text{cm}^2 \sim 25\text{w}/\text{cm}^2$ ) .
3. Click “start”, put the service brake under your feet, step on to start working, move away for pause.
4. Beautician hold the ultrasound head to move slowly on skin, in circle or beeline repeatedly, the other hand push the fat towards the ultrasound head.
5. Do not use on back, avoid using on bones.
6. Do not use on uterus part when women do abdomen care. Abdomen treatment during menstruate should be avoided.
- \* Enough gel are needed, you may feel painful if medium is not enough.
- \* The total treatment time for daily care should be within 30 min.
7. The machine will pause automatically if the setting time is over.
8. Clean the leftovers with hot towel, use warm towel to clean the ultrasound head.

## Caution

1. There should be enough gel when using the ultrasound head.
2. The ultrasound head should not stay out of work for too long time while outputting energy. Click "pause" to stop working to avoid damage.
3. M80 ultrasound heads should not stay still at one part for too long time, and do not use it on bones.
4. Please step on service brake then there will be sound wave exist when using M80 ultrasound head.
5. M80 ultrasound head can not be used on back, avoid on the bones, and daily care should be within 30 minutes every time.

## Vacuum&roller treatment

1. Customer put on special slimming clothing after taking a shower.
2. Operation-select fat rolling function, cling the big roller to the slimming clothing.  
Vacuum mode: P1~P9—pulse suction  
P0—continuous suction  
Fat rolling mode: -B1(pull back towards)+B2(push forward)+B3(roll inward)  
Fat rolling speed: 1(slow)~10(fast)  
Suggestion: Decompose fat, burn fat- B3+pulse suction+ red light  
Lymph de-toxin, edema elimination- B2+pulse suction+ light combination  
Slimming and body shaping, skin tightening- B2+continuous suction+ red light  
Cellulite elimination, massage relax- B1(B2)+pulse suction+ blue light
3. Choose the fat slip function, use vacuum& small roller, fit for breast, arms, crus, thigh, cervix.

## Caution

1. Do not eat two hours before treatment.
2. The customer needs to put on special slimming clothing, do not use any products.
3. To regulate vacuum flow according to the guests' tolerance, from weak gradually to strong.
4. Vacuum & big roller suit for thick muscle, fleshy part, or substantial subcutaneous tissue. The thin, uneven parts not fit for use. Do not on head, face, heart and other parts of the operation.
5. As people have different skin and health status, there may be red mark after operation, this is a normal phenomenon, may subside on its own.
6. Customers need to drink eight glasses of water every day (about 250ml/cup) to help drain toxins away.
7. The filter must be replaced after using for 40 hours (cumulative), or it will affect the suction.

Period of treatment: 20 times/a period of treatment    3 times/week    30~60min/times

Change the filter: open the lid as the arrowhead indicates, take the filter out, and change a new one, fasten the lid in the opposite direction of the arrowhead.

Notes: filters of big (small) roller should be changed every 40 hours



## Technique parameters

Voltage:110-230V/50/60Hz

Vacuum Moment:5000g.cm

Host Power $\leq$ 1000W

Vacuum $\leq$ 10h/m<sup>3</sup>

Strong ultrasound:40KHz

Fiber $<$ 5Mw

## Accessories list

1.Host base	1set
2.Host upside	1pc
3.Power supply line	1pc
4.Use manual	1pc
5.M80 ultrasound head	1pc
6.Service brake	1pc
7.Host upside connect screw (M4*16)	2pcs
8.Upside accessories screw (M6*12)	2pcs
9.Hexagon screwdriver (M3,M5 each 1pc)	2pcs
10.Filter	2pcs
11.Slimming clothing	1set(5pcs)
12.Airproof circle (8*1.9)	4pcs
13.Airproof circle (19*2)	4pcs

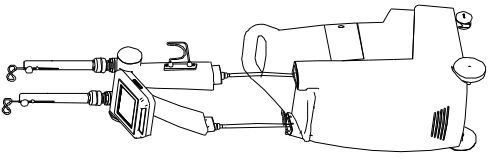


Fig 4

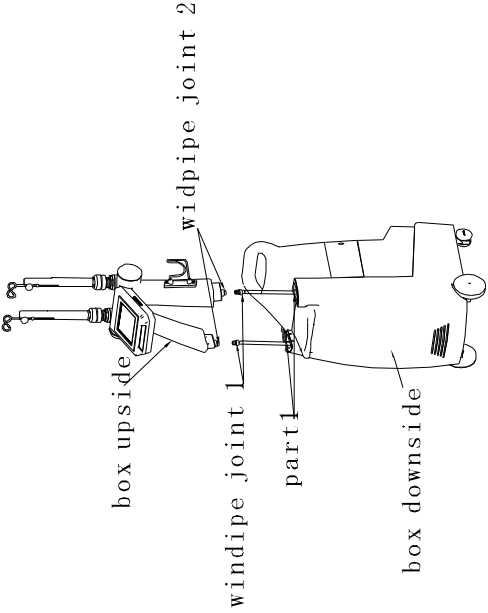


Fig 3

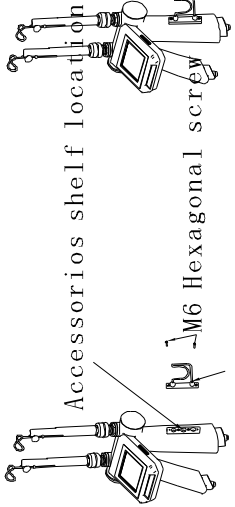


Fig 1

Assemble step 1

1: Assemble accessories shelf at right place aim at the holes of the box

2: Fasten the accessories shelf to box with M6 screws as Fig 2

Assemble step 2

1: At part 1, to bring up windpipe joint 1, to connect to windpipe joint 1

2: To fasten windpipe joint 1 to windpipe joint 2, as fig 4

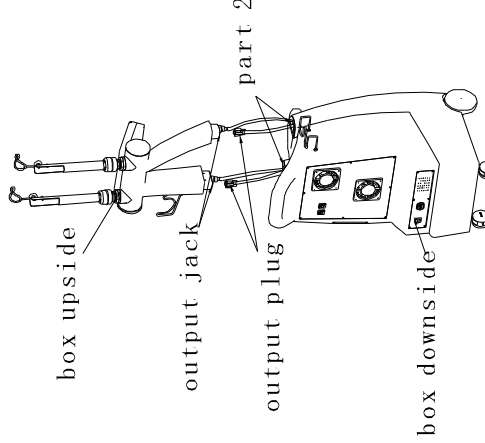


fig 5

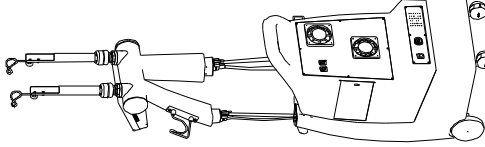


Fig 6

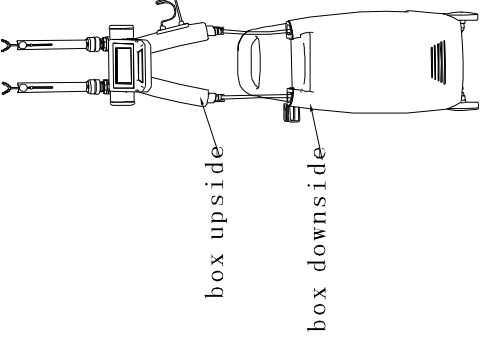


Fig 7

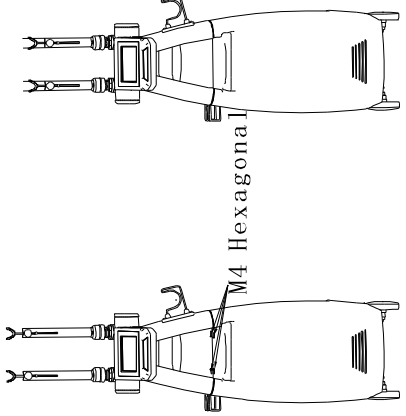


Fig 8

Assemble step 3

1: At part 2, to bring up output plug at downside of the box, insert it into output jack of the upside of box

2: Fasten the screws of the plug as fig 6

Assemble step 4

1: To connect box upside to box downside

2: Fasten with M4 Hexagonal screws, finish

Finish