

LN-LS653

Laser Fat System

User manual



Warning

Dear user:

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:



- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

This product does not belong to medical, please don't use for disease treatment!

Parts Introduction



- 1.LCD screen
- 2.LEMO wire commutator jack2
- 3.LEMO wire commutator jack3
- 4.LEMO wire commutator jack4
- 5.LEMO wire commutator jack5
- 6.General power switch
- 7.Power supply line socket
- 8.Laser 2X work head jack
- 9.Laser20X work head jack

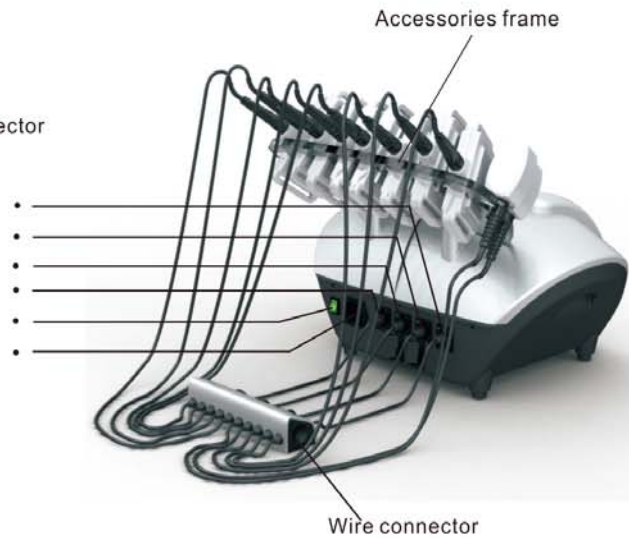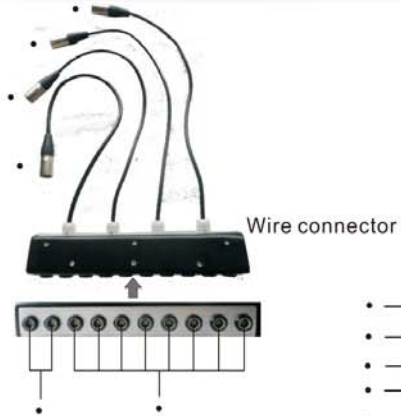

Accessories



Laser 2X work head




Laser 20X work head

Basal Operation

- Put the accessories well and connect them to the host.
 - Insert LEMO wire commutator 2,3,4,5 to jack [2],[3],[4],[5];
 - Insert two Laser 2X work head to [8];
 - Insert eight Laser 20X work head to [9];
- Insert power supply line to [7], turn on the General power switch [6].
- As (Fig 1) shows, input default Password "000000" to enter interface as (Fig 2).
- Click any place on screen to enter function interface as (Fig 3) shows.



(Fig 1)

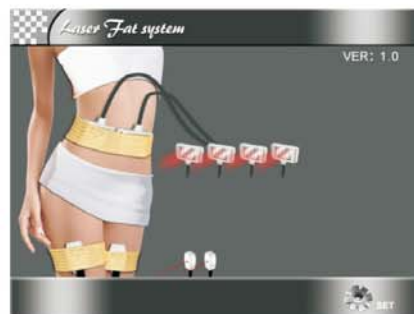

(Fig 2)

Laser Function

1.Operation step of Laser 2X

- Clean the skin, and according the treatment part to choose the suit band ties and tied them up.
- Put the Laser 2X work heads into the band ties, the side with Led contact to skin directly.
- Set intensity of every Laser 2X in interface (Fig 3) shows. Click "←" and "→" to set, the range is 1(weak)~3(strong).

Operation steps:

When the cursor "V" on the location "channel 1 1Hz", click "←" and "→" to set the intensity of laser, the range is 1HZ~1000HZ. OFF means that close channel 1.

Then move the cursor "V" to the "channel 2 1Hz", click "←" and "→" to set the intensity of laser, the range is 1HZ~1000HZ. OFF means that close channel 2.

- Setting work time

Click "Time" to enter interface (Fig 4). Click "←" and "→" one time to change one minute, click the number of time directly to change ten minutes every time.30~45 minutes of every treatment part.

- click "⏏" to start work.



(Fig 3)



(Fig 4)

(6)The machine will pause automatically if the setting time is over. Beautician take the 2X Laser work heads out and put them well, then loosen the band ties.

(7)Click " " to return back interface as (Fig 2) shows.

2.Operation steps of 20X Laser work head

(1)Click the "Laser 20X" in interface (Fig 3) shows to enter interface (Fig 5).

(2)The other steps please refer to "Laser 2X Function".



(Fig 5)



CAUTION LASER: Do not let the laser light straightly shine upon the eyes in Case of eyes hurt.



(Fig 6)

Others

Click "set" as the(Fig 1) shows. Enter interface as (Fig 6):

Click" basic setting ", enter the selecting language interface.

Notes "advanced setting" is for manufacturer only.

Forbidden group

- 1.People with heart disease or high blood pressure, or configured cardiac pacemaker.
- 2.Patients with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
- 3.People with hemorrhagic disease, trauma or who is bleeding.
- 4.Pregnant women
- 5.Medical Plastic parts, or parts with artificial in filling inside.
- 6.People with metal inside the body(include metallic tooth)
- 7.Patient with skin inflammation or with edema.
- 8.People with an abnormal immune system.
- 9.Numb or insensitive to heat.

Technical parameters

Input: AC110V/60Hz ☐ AC220V/50Hz☐

Power: $\leq 50\text{VA}$

Accessories list

1.Host	1set
2.Power supply line	1pc
3.User manual	1copy
4.Laser 2X work head	2pc
5.Laser 20X work head	8pc
6.Bind ties	1set
7.Accessory frame	1pc
8.Hexagonal screws	2pcs
9.Hexagon wrench	1pc
10.Wire connector	1pc