


Cryolipolysis freezing slimming machine

Cryolipolysis+40K cavitation+RF

Manual

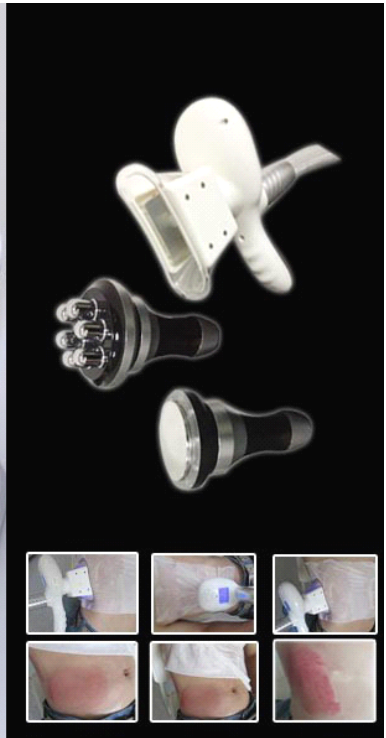

CONTENTS

Warning.....	3
Technology background.....	4
Part Introduction.....	5
Annexes Function and Treatment Principles.....	6
Cryolipolysis handle.....	6
40K cavitation handle.....	7
Body RF handle.....	8
Operation Instruction.....	9
Important Safety Notice.....	10
Safety Notice.....	11
Main technical parameter.....	13
After-sale service and maintenance.....	14

Warning

Dear users:

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggest as follows:

Please read this instruction detailedly, and keep it well for further reading and consulting.

Please follow the structure fit and operate the machine correctly.

Please do not remove and change any accessories of the machine.

It forbids other to open and remove the mainframe except the authorized people in the company.

It forbids others to change and maintain the machine except the professional people.

Please contact us if there is any trouble during operation.

This product does not belong to medical, please don't use for disease treatment.

Technology background

Too much fat can damage people's appearance and athletic ability. May also cause various diseases. Too much fatty tissue can be located in various parts of the body, including the thigh, buttocks, abdomen, knee, back, face, arms and other areas. And an excess of adipose tissue may be cause ugly appearance formed by cellular. Excess body fat also related to significant health risk. So we need effective method to control or eliminate excess body fat.

Liposuction is a cosmetic surgery procedure that slims and reshapes specific areas of the body by removing extra fat deposits. Usually operated by the Plastic Surgery or dermatologist with a special surgical instruments for liposuction, it is an invasive way to remove subcutaneous fat tissue.


There are also possible fatal complications and negative side effects associated with having liposuction. These temporary factors include swelling, painful bruising, pain, numbness, limited mobility and allergic reaction. In addition, liposuction is of high cost. There are other new emerging technologies to remove subcutaneous adipose tissue, including Mesotherapy laser-assisted liposuction and high-intensity focused subcutaneous adipose tissue, including Mesotherapy laser-assisted liposuction and high-intensity focused ultrasound

Other non-invasive way to remove excessive fat including drugs, diet pills, regular exercise, diet control, or a combination of these methods. A disadvantage of these treatments is likely invalid, and even in some cases impossible. For example, when a person physically injured or sick, you can not select the way of regular exercise. Similarly, when the diet pills or external use of drugs are allergic or cause adverse reactions, in addition, the use of physical or systemic approach to lose weight, you can not eliminate the body fat of a selected area.

The Cooling procedure using Cryolipolysis is fundamentally different from other non- or minimally- invasive modalities, and it was approved as the most effective way to fat reduction.

This revolutionary new fat loss method, which is approved as an important technology progress in fat reduction since it is put into use. For those people who is on diet, do regular exercise but still remove local fat, Cryolipolysis definitely a great gift. For fat intensive parts and less parts, such as love handles (flanks), and back fat. (loose fat above the hip on both sides of waist), belly and back fat, Cryolipolysis can produce stunning results.

Part Introduction

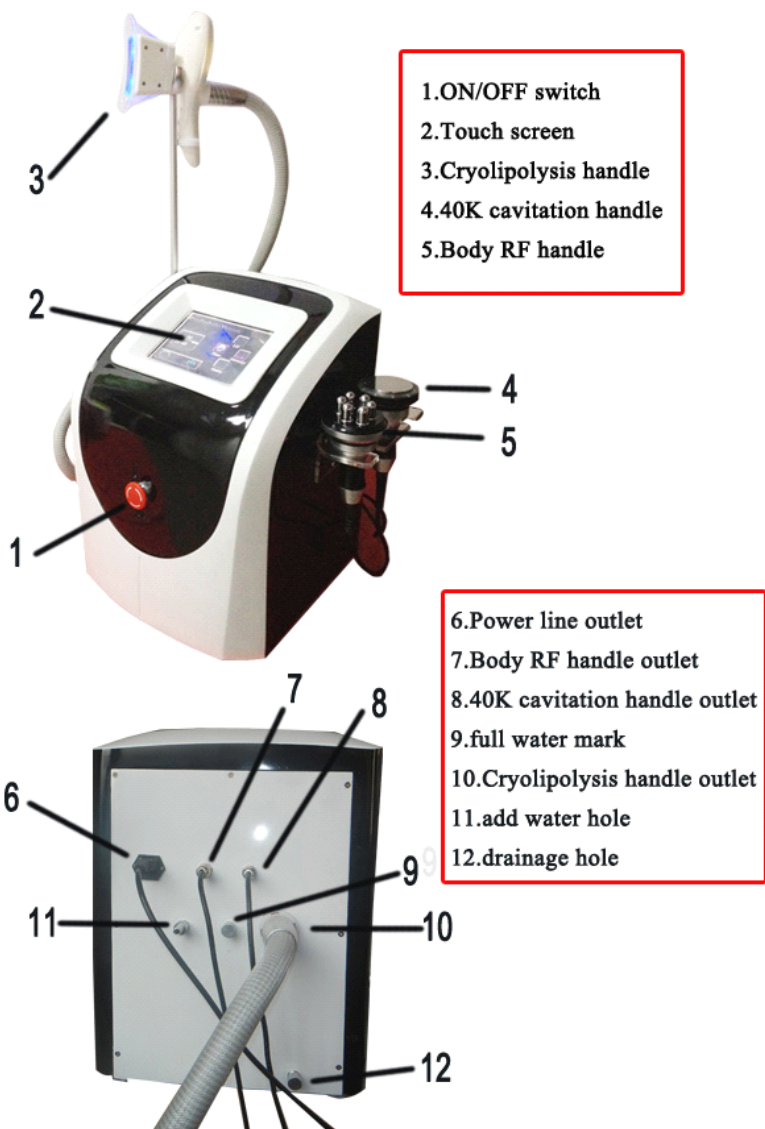

Annexes Function and Treatment Principles

1.Cryolipolysis handle



As triglyceride in fats will be converted into solid in particular low temperatures, It uses advanced cooling technology to selectively target fat bulges and eliminate fat cells through a gradual process that does not harm the surrounding tissues, reduce unwanted fat, When fat cells are exposed to precise cooling, they trigger a process of natural

removal that gradually reduces the thickness of the fat layer. And The fat cells in the treated area are gently eliminated through the body's normal metabolism process, to eliminate unwanted fat.



This handle has its own switch,Use this handle you need notice:

- 1.while Insert cryolipolysis handle,put it straight,so the water can circulate.
- 2.turn on the switch and wait it cool
- 3.prepare antifreeze



When feel the hot head,Turn off the switch immediately and contact with us.



2.40K cavitation handle:



Strong sound wave explosion fat head with collective strong sound wave head, strong sound wave of 40KHz may be emitted to human body for impacting fat cells fiercely and causing friction motion between fat cells. This may cause effective consumption of calories and moisture in fat cells

and reduce the size of fat cells. what's more, sound wave vibration may cause fierce impact of fat cells to make them be exploded instantaneously, reduce the amount of fat cells and thereby achieve the effects of removing fat.



3.Body RF handle



Radio frequency is one kind of high frequency electromagnetic wave, it can penetrate the dermis and subcutaneous tissue through the fat layer, take heat into deep skin and then accelerate regeneration and reconstruction of skin deep-tissue.

RF can produce energy by impedance from the skin, which can make the fatty layer temperature increase in very short time, and this heat will also act on collagen in the dermis layer. Collagen in the dermis layer will shrink immediately after the temperature rise and proliferation of new collagen, so as to achieve the effect of skin pulling, pigmentation improvement, skin whitening, and wrinkle removal.



Operation Instruction



Step1:

Open the water knob


This machine is use the top-class dual channel water circulate refrigeration science.so before you use,must add water in.



Step2:


Put pure water in

use the add water system(water pipe,funnel,filler) ,put about 2L pure water in.

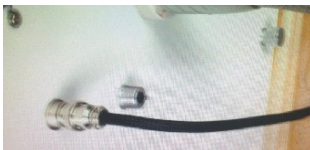

Step 3:Fill up with water

Add water until the water out of the full water mark.

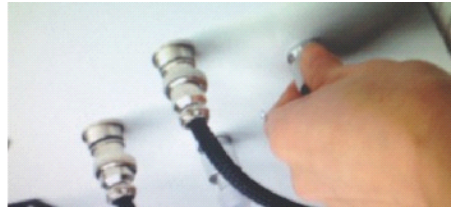

Step4:Tighten the knob



Step 5:

Place the fittings well

make sure the fittings are connected closely,and do not affect them working.

Step 6:Turn on the machine switch

Step 7: Select handle begin work (the operating instructions details and notice see the Annexes ahead)

Important Safety Notice

- I. Please don't disassemble the equipment or try to do some other operation, which haven't been indicated in our instruction. All the repairing work should be done by our admitted professional personnel
- II. Please don't assemble or operate this equipment while you are near the water or your hands are wet. Notice that don't pour the liquid on it.
- III. Please pull out the plug from the socket when you met such conditions as below. And in touch with professional maintenance Station.
 - A. The equipment touched liquid
 - B. The equipment has abnormal smell, smoking or loud and strong noise.
 - C. Cable is broken
 - D. The equipment is fallen down and broken
- IV. Please don't put anything on the cables, don't place the equipment at any place where someone can step on the cable.
- V. Please pull out the power cable after using it for your safety
- VI. Don't put anything into the equipment's jack, otherwise it may cause fire or touch the electricity. If something enters the equipment, please don't pick it out by yourself, contract distributor or our company immediately
- VII. Please don't put the equipment on the anything that can't stand steadily, avoid it turns over and damage.
- VIII. Our company has the right to modify the information in this introduction before notify! And keep the explanations right for the above.
- IX. **Never use the strong acid or strong base or alcohol clean the machine.you can take physiological saline.**

Safety Notice



Before operation, please read the following content.

1. It is essential to make different setting according to specific treatment demand.
2. The application of The machine must never be painful and cause redness to the skin or leave marks.
3. Move the treatment heads slowly over the skin of the body without ever letting them stop in any one place on the skin.
4. Do not apply the device on children less than 7 years old. The treatment to children over 7 requires medical advice.
5. Do not use this device if suffering from a fever, inflammation, dermatitis (acne, eczema, herpes), warts, open or recently stitched wounds, serious and progressive illnesses (cancer, AIDS, hepatitis, blood diseases, etc. or blood coagulation problems. Do not apply the device if you have or have had phlebitis, after veins surgery within the last two months, if you have varicose veins or tingling in your limbs, if you have any concerns consult a physician.
6. Never use the device on a pregnant woman.
7. May be used on all external and healthy areas of the body except the eyes, throat, ears, glottis and the genital area.
8. Must not be applied inside the mouth, or on mucous membrane, or on nipples, or moles without medical advice.
9. Never use the device if you suffer from epilepsy. The device must not be used by any person suffering from mental illness.
10. In the case of long-term treatment with drugs such as anti-inflammation, anticoagulants, antibiotics or if your skin bruises at the slightest shock, ask advice of your doctor/physician before using.

11. Never apply to the body parts with jewelry or ring.
12. Regular use of The machine enhances well-being; it is better not to stop its use for a prolonged period of time. Not following operating instructions and usage recommendations may bring lesser results than expected. Do not use the device longer than the recommended time.
13. If your symptoms have no improvement, or if in any doubt on the use, consult your doctor.
14. Effect may vary depending on your age, life habits (level of physical activity, diet, etc.).
15. The machine can be used after any plastic surgery only while following the instruction of the surgeon.
16. The machine is designed for professional use only, any personal use of this device is prohibited.
17. The Machine continuous working time shall not exceed 2 hours. The best way is cut the power off, let the machine have a rest for 10-15 mins. After the machine have worked for a hour.

9. Treated scope

abdominal fat, love handles, back rolls, muffin-tops and bulges above Cesarean section incision lines

MAIN TECHNICAL PARAMETER

Voltage AC110V 50

Power <300W

Temperature: -14 °C~14°C

Pressure output :0-900Kpa

Screen:8 inch touch screen

Gross weight:30 KG

paking size: 68cm*52cm*49cm

ACCESSORIES LIST

No.	Description	Qty.
1	Cryolipolysis handle	1Pc
2	40K cavitation handle	1 pc
3	Body RF handle	1pc
4	Anti freeze mask	10pcs
5	Power cord	1pc
6	User manual	1book

AFTER-SALE SERVICE AND MAINTENANCE

After-sale service

We will guarantee the quality of our equipment, normally there is no quality problem

1. If there is a deficiency, please email or call us on the reasons, it would be best if this can be dealt with by email or telephone. Because some of the problems are caused by using the machine in a wrong way.
2. We suggest the client to take photos of the area where the problem occurs, we will pass them to our technician , our technician will give a satisfactory answer within 1-7 working days
3. If the problem comes from the spare part, we could consider in changing the spare part for our client, which could be sent with his/her next order. If the next order is uncertain, we could pay freight cost to send the spare parts.
4. If the problem comes from the main device, we could advice our client to post the device back to our company for repair by ordinary post, or we could re-send a new.