

H200 Water oxygen sytem

Manual



Warning	1
Control panel & parts introduction	2
How it works	3
Operation	3
Main effect	4
Technical Features	4
Technique parameters	4
Annex List	5

Warning



Dear user:

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:

- ◆ ◆ Please read this instruction in detail, and keep it well for further reading and consulting.
- ◆ ◆ Please follow the struction to fit and operate the machine correctly.
- ◆ ◆ Please do not remove and change any accessories of the machine.
- ◆ ◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆ ◆ It forbids others to change and maintain the machine except the professional people.
- ◆ ◆ Please contact us if there is any trouble during operation.

This product does not belong to medical, please don't use for diease treament!

Control panel & parts introduction

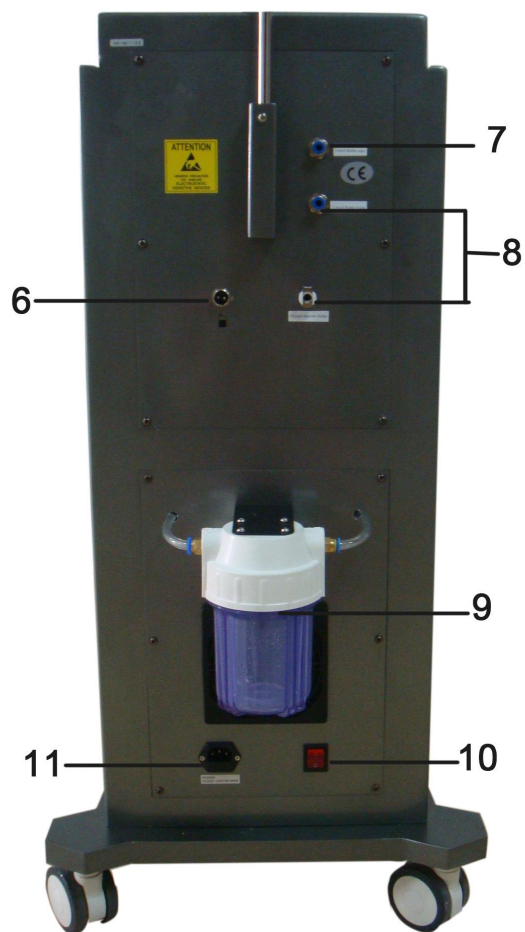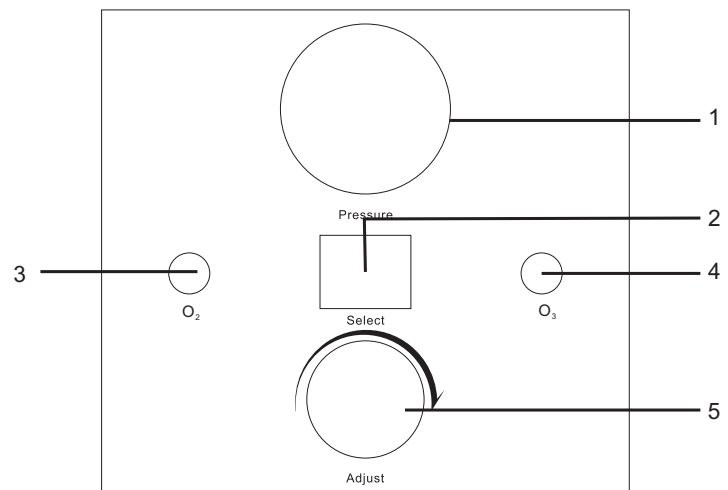

- 1.Barometer
- 2.Start/pause
- 3.O₂ indicate light
- 4.O₃ indicate light
- 5.Air pressure adjust valve
- 6.Foot switch outlet
- 7.Liquid bottle outle
- 8.Oxygen injector outlet
- 9.Filter recyclingbottle
- 10.Generalpowerswitch
- 11.Powerlinejack

How it works

Use of hyperbaric oxygen, push the sterile saline (saline) and oxygen access air compressors simultaneously, transfer water drops of the particle type into small molecular (5-200μm microns), tiny water- drops would spray (jet spray), with 200m/sec (seconds) speed (supersonic) sprayed by a specially designed nozzle, inject water and oxygen into the deep skin speedy. Exfoliating and do needle-free nutritional delivery treatment painless, noninvasively.

The impact generated by hyperbaric oxygen and saline, physical exfoliating procedure suit for any skin, it can stimulate the cells regeneration function, stimulating blood circulation, strengthen the metabolism of skin cells and collagen proliferation!

Operation

1.connect the handles to the host.

(1) Insert foot switch to [6].

(2) Connect one end of the 1# transparent tube to the liquid bottle,(there should be liquid product inside),the other end to [7].

(3) Insert 3 tubes of the oxygen injector into [8] correctly.

2.Connect power supply line [10],turn on power switch.

3.Choose output mode,turn the start button[2] to left, “I” is O2 output,[3] the indicate light is on.turn the start button [2] to right, “II” is O3 output, [4] indicate light is on.

4.Take off the metal protective cover of the oxygen injector,press the foot switch is working, release is to stop output.

5.Turn [5]air pressure adjust valve, the air pressure is suitable from 40psi-60psi, when the pressure is over 80psi,the machine will alarm.



Increase flow



Increase flow

6.You can use the switch valve in the transparent tube to control the flux.

7.The flux is maximum when the angle between the switch valve and the tube is 90°,when the the angle between swith valve and the tube is 180°,it means to shut up the flux.

8.If need to stop injecting during operation,you need to shut up the liquid output immediately. press [2] to stop,and turn off [9] general power switch.

9. When the liquid is up to the warning line of the [11] Filter recycling bottle, you need to clear the bottle in time, the clockwise direction is removed, counterclockwise is to screw.

Main effect

1. Removal of scar: burn, surgery scar; acne pits, Pockmark and so on.
2. Removal of acne: remove acne and inflammation, acne, blackheads, acne and acne scar.
3. Skin rejuvenation: remove splash, Pigment deposition, shrink pores; whiten skin; enhance skin elasticity and luster; improving skin all over the body.
4. Wrinkle removal: Lips wrinkles, eye wrinkles; improve glabellar lines, stretch marks and so on.
5. Hair growth: improve seborrheic alopecia, alopecia areata; hair care.
6. Accelerate blood circulation, increase skin elasticity, reduce skin relaxation.
7. Repair damaged skin

Technical Features

1. Non-invasive physical scrub rejuvenation technology
2. No need to configure the dangerous oxygen bottles, it can also achieve the 6-8Kg pressure
3. Unique design of the sprayer with strong pressure
4. With air compressed fluid recycle and refrigeration plant.
5. Equipped with pulse oxygen spray technique.

Technique parameters

Voltage: AC110V/220V 50-60Hz

Power: < 750W

pressure: 0-8kg

Annex List

1.Host	1pc
2.Power line	1pc
3.User manual	1pc
4.Foot switch	1pc
5.Accessories support 1	1pc
6.Stainlesshexagonalscrew	4pcs(m6*16mm)
7.Steelpipe	1pc
8.Pothook	1set
9.Bottle	1pc(withsockettotheconnector)
10.Sprayer	2pc
11.Plug+tube	2pcs(vavle 1pc)
12.Tube(with connector)	1pc(vavle 1pc)