

2012!!!

Hydradermabrasion Facial Rejuvenation Machine

(Instruction)



IHSPA7.0


Accessories List

1. Host	1pc
2. Hydra handle	1pc
3. PVC Hydra tips	2sets(16pcs)
4. Emery Hydra tips	5pcs
5 .500ml Liquid bottles	1pc
6 .Wastewater recycling bottle	1pc
7 .Power supply line	1pc
8. Accessories shelf	1pc
9. User manual	1pc
10. Hexagonal screws	6pcs
11. Hexagon screwdriver	1pc
12. Seal Circle of vacuum Spa handle 1.2*2mm	2pcs
13. Water regulating valve	1pc

Catalogue

Parts Instruction.....	1
Accessories.....	2
General Introduction.....	3
Operation.....	3
Effect.....	5
Caution.....	5
Forbidden group.....	6
Technical parameters.....	6
Accessories list.....	7

Parts introduction

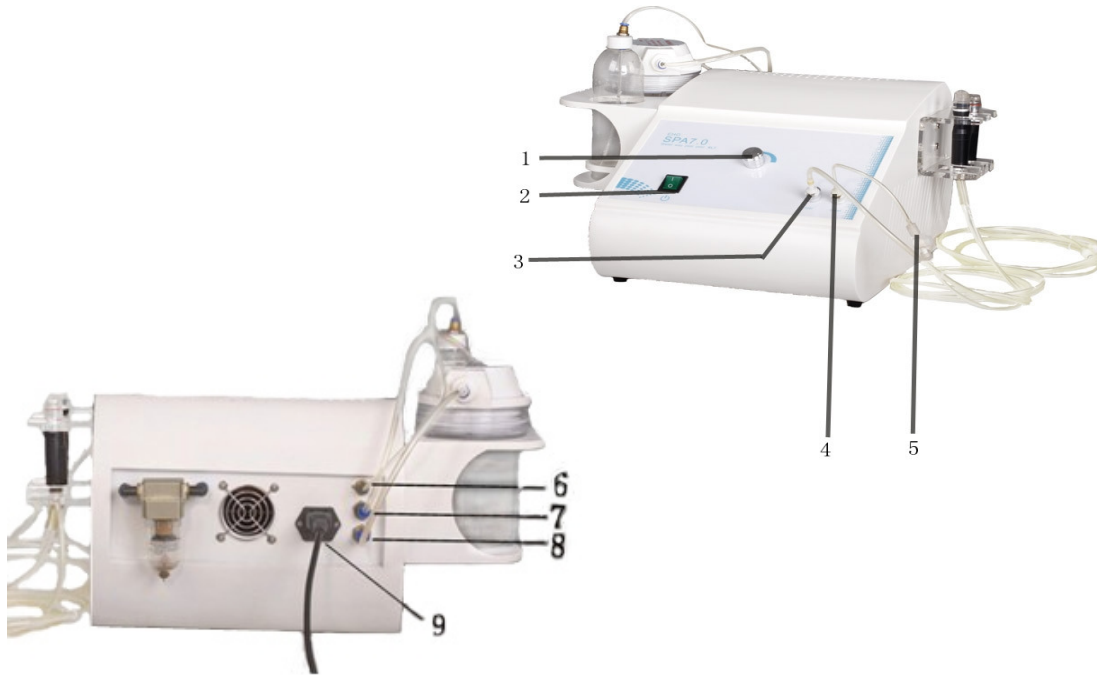

1. Vacuum debugging switch
2. General power switch
3. Hydra intake handle
4. Hydra outlet handle
5. Water regulating valve
6. Liquid bottles intake handle
7. Intake handle for waster water recycling bottle
8. Outlet handle for waster water recycling bottle
9. Power supply line

Forbidden Group

1. People who has fever, infectious diseases, acute diseases.
2. People with hemorrhagic disease, trauma, vascular rupture, skin inflammation, skin disease.

Technical Parameter

Input: 220V / 50HZ ☐ 110V / 60HZ ☐

Power: <80VA

Output pressure: 0-100KPa

Effect

1. Use the new technology of hydra liquid water
2. Deeply dermabrasion of the skin
3. Suitable for all skin types
4. Ranked above IPL for skin rejuvenation
5. Provides non-invasive, non-surgical procedure options
6. Show immediate results without discomfort or downtime
7. Targets specific skin concerns such as sun damage, fine lines and wrinkles, dehydration, and oily/dehydrated skin

Caution

1. Do not use on wound.
2. Pour out the water in the waste recycling bottle in time, to avoid damage caused by overflowing.
3. Different size of vacuum work head can be flush with water directly, Then soaked with 75% alcohol.



Diamond Hydro peel handle



Vacuum SPA handle




Recycling bottle



Liquid bottle

Vacuum SPA Heads

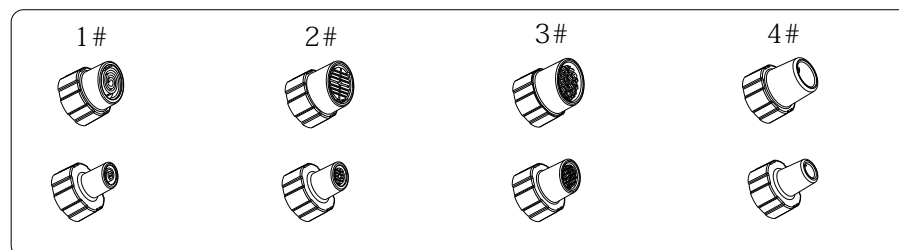


General Introduction

The Hydra-dermabrasion resurfacing treatment thoroughly enhances the skin using vortex technology to cleanse, exfoliate, extract, and hydrate simultaneously. It merges soothing and invigorating spa therapies with advanced medical technology to achieve instant lasting results. The procedure is soothing, moisturizing, non-irritating, and immediately effective. The treatment is clinically proven to improve the appearance of fine lines, wrinkles, congested and enlarged pores, oily or acne-prone skin, hyperpigmentation and brown spots.


Operation

1. Install the annex on the host, place all the annex and connect with the host:
 - (1) Insert the vacuum Spa intake handle into [3].
 - (2) Insert the vacuum Spa outlet handle (with Water regulating valve) into [4].
 - (3) Install the liquid bottles intake handle into [6].
 - (4) Install the wastewater bottles intake handle into [7].
 - (5) Install the wastewater bottles outlet handle into [8].
 - (6) Install the Power supply line into [9].
2. Put appropriate liquid into the bottle
3. Choose the hydra tips (pvc or emery), according to different treatment, and connect to the hydra handle.
 - 1# tips for clearing comedones, Whitehead and acne
 - 2# tips for deep cleaning of normal skin, uneven color and dark skin, dry aging skin
 - 3# tips for thick cuticle, coarse pores, oil over secretion skin.
4. Open the power switch [2].

5. Adjust the Hydra debugging switch [1] (left - Small, right - large), transferred into a suitable pressure.
6. Beautician hold the hydra handle on skin for a while until the water flow into the tips, and then drop some water on skin by points, to make the skin surface wet, and then apply cleansing product on the skin, and move the tips on skin back and forth.
7. Adjust the water regulating valve [5], the instruction as follows:

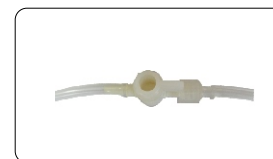

Opened

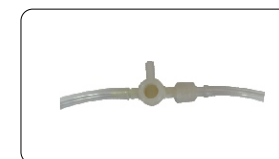

Closed

8. Check the Wastewater recycling bottle [7] before turn on the Machine, if the bottle is full, please clean it.

Cleaning Method:

Unscrew the wastewater recycling bottle in a anticlockwise direction, cleaning the wastewater, then fasten it in a clockwise direction.