


# LN-G6

## Manual





## Parts narration

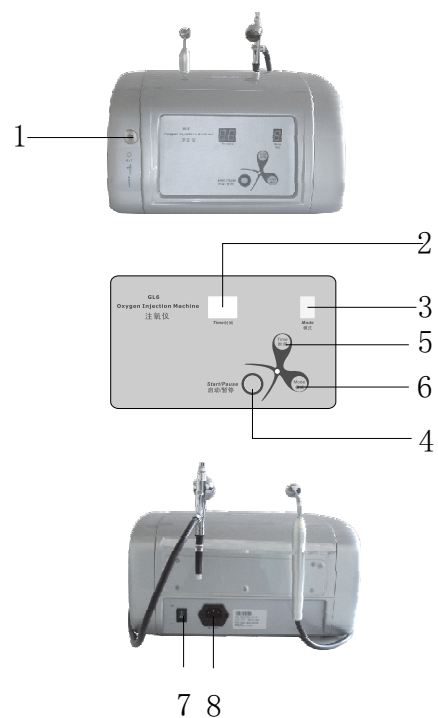

- 1.Output
- 2.Work time indicator
- 3.Work mode indicator
- 4.Start/pause
5. Work time regulator
- 6.Work mode regulator
- 7.General power switch
- 8.Power supply line jack

## Accessories



Oxygen injector



Oxygen sprayer

## Operation

- 1.Insert oxygen sprayer or oxygen injector into[1]
- 2.Connect power supply line, turn on general power switch
3. Press[5]to set work time(1:00~60:00min, every press means increase 5 min)
- 4.Press[6] to regulate work mode  
oxygen sprayer:1—continous spray oxygen  
oxygen injector:1—continous spray oxygen  
2~3—Intermittent inject oxygen
- 5.Usage of oxygen sprayer  
Pour essence or nutrition liquid into small cup of the oxygen spray gun, 2/3 of capacity is better, press [4]to start the machine,press the control button of the sprayer and pull backwards,then it will spray out. To start from the forehead to the neck when spray oxygen,to handle with one hand,the other do massage, lifting along the muscle.the speed should be regulated according to absorb condition of the customer(about 20 min)
- 6.Usage of the oxygen injector  
Press [4] to start machine,cling the oxygen injector to skin and move slowly and in order.  
Forehead - right eye - right cheek - Chin - left cheek - left eye - forehead (about 20 minutes)
- 7.If the setting time is over, machine will suspend automatically.
8. Operation finished.turn off the power,unplug the power supply line.

## Caution

- 1.The product should be liquid when using oxygen sprayer.
- 2.Do not spray into mouth or nose.
- 3.Clean the small cup and the pipe inside the sprayer everytime use it, to avoid jam. Clean: pour the purified water into the small cup, and press the control button and then pull backwards.
4. If there need a suspend, press [4] pause.

## Technology Parameter

Voltage: 220V/50Hz Power: ≤75W

## Accessories list

1.Host	1pc
2.Power supply line	1pc
3.Use manual	1pc
4.Oxygen sprayer	1pc
5.Oxygen injector	1pc
6.Accessories shelf	1pc
7.Hexagonal screw M3*6	3pcs
8.Hexagonal screwdriver M2.5	1pc