SPA9.0

Hydra facial Dermabrasion Machine INSTRUCTION




Dear customer,


Thanks you very much for choosing our product!

For the best capability and safety, please read the instruction before operation.

Please reserve the instruction book for the reference.

Introduction of panel and accessory

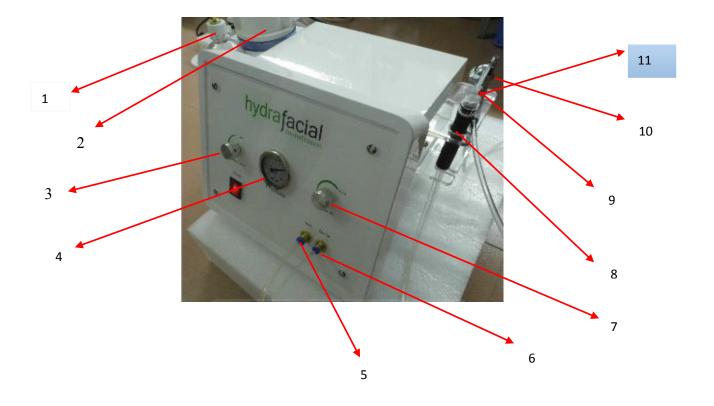

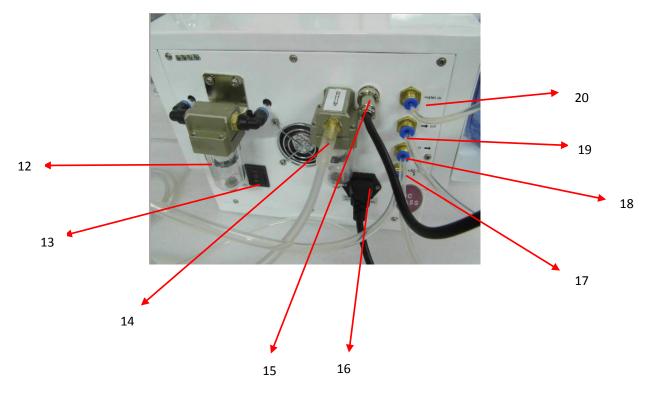

1. Clean water bottle

2. Waste water bottle

- 3. Water flow adjust valve
- 5. Water out
- 7. Vacuum valve
- 9. Oxygen spray handle
- 11.Ultrasonic skin scrubber
- 13:Effect switch
- 15:auto alarm
- 17:oxygen spray connector
- 19:waste water out

- 4. Oil pressure gauge
- 6. Water in
- 8. Hydra handle
- 10.Micro-dermabrasion
- 12:dead skin
- 14:Micro-dermabrasion connector
- 16:Power switch
- 18:waste bottle water in
- 20:Clean water in

Hydra facial dermabrasion functions:

- 1)hydra facial
- 2)Diamond micro-dermabrasion
- 3)Oxygen spray
- 4)Ultrasonic skin scrubber

Effect:

Hydra facial dermabrasion help to improve the appearance of

- -fine lines & wrinkles
- -hyperpigmentation & melasma
- -sun damaged skin
- -age spots
- -blackheads & whiteheads
- -enlarged pores
- -excess oil
- -hydration
- -puffy eyes
- -rejuvenation, overall skin health

Operation skills:

1)diamond micro-dermabrasion




1: pipe and diamond tips connect to handle and then turn on the diamond peel switch




Diamond peel switch


Step 3: put diamond wand tightly onto corresponding skin according to different tips

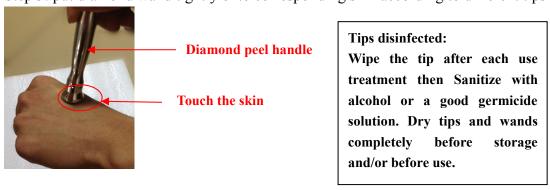


2)Hydra facial dermabrasion

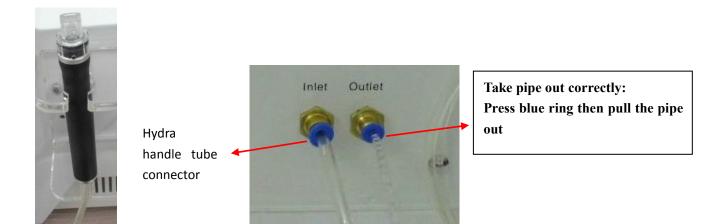


2: PVC tips connect to aqua peel handle

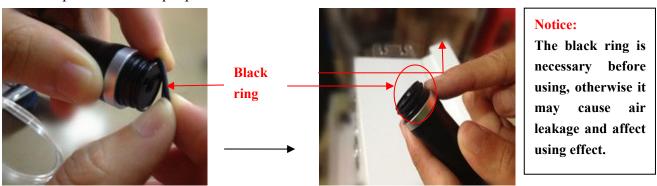

3: put essence into clean water bottle and then connect clean and recycle water bottle to mainframe

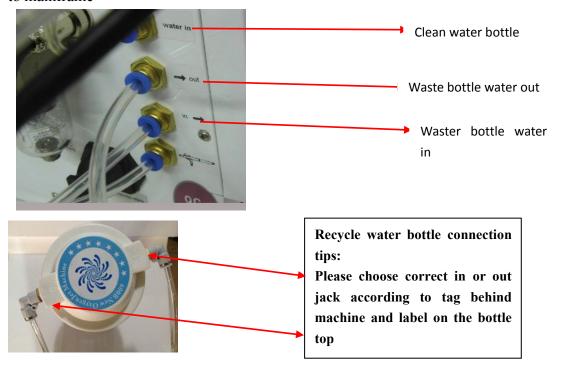

About waste water bottle installment:

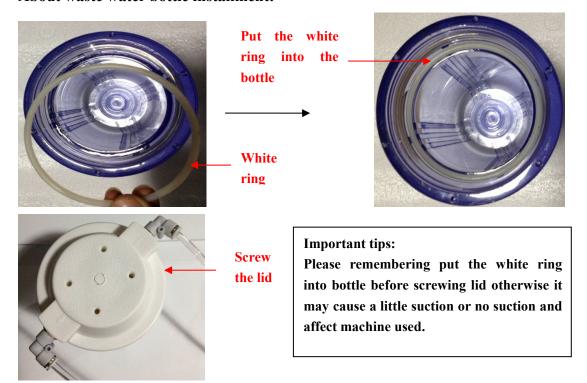


3)Oxygen spray




4)Ultrasonic skin scrubber




1:connect pipe to spray and turn on the machine

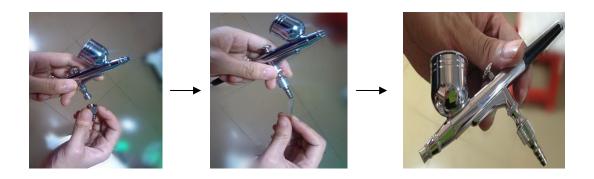

2: connect the spray on the back of the machine

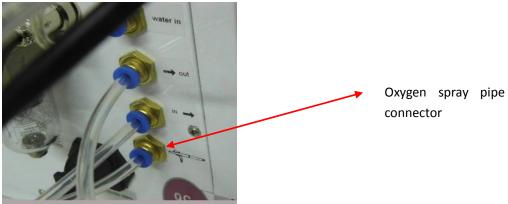

3:add 2-3ml serums/lotion to spray

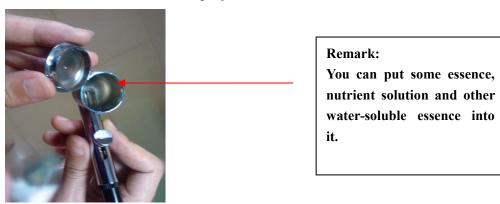

4: start spray and you can see the nice effect of water/serums spray



Technique parameter

Electricity voltage: AC90V-220v

Working frequency: 50Hz/60HZ

Power: 250w

Packing size: 73*45*40cm

Gross weight: 25KG

Accessories list accessories

Main body	1pc
Clean water bottle	1pc
Waste water bottle	1pc
Hydra handle	1pc
Hydra facial tips	8pcs
Micro-dermabrasion handle	2pcs
Micro tips	9pcs
Oxygen spray handle	1pc
Black rings	3pcs
Ultrasonic skin scrubber	1pc

Solution for simple failure

Failure	Solution
Power on, main machine no response	Check power connection
	Check whether power source is on
No output	Check the air pipe connection
During operation, instrument is overheat	Using for too long time, need to cool down in a nature way
	Check whether voltage is right (need professional technician to check)
No power	Check whether fuse is burnt, if so need to replace a new one