
Crystal&Diamond Dermabrasion Apparatus

(T12)

Instructions



Contraindication

- 1 Who has sensitive skin, tumescent or inflammation symptom.
- 2 Patient who has bleb, wart, skin disease.
- 3 Diabetic
- 4 Immunity maladjustment

Accessories list

1. Host			
2. Power supply line	1pc	1 set	
3. User manul		1 copy	
4. Transparent tube		1pc	
5. Crystal dermabrasion cap	5pc		
6. Diamond dermabrasion handle	1pc		
7. Diamond dermabrasion rob	1box (3pc)		
8. Diamond dermabrasion head	1box (10pc)		
9. Cotton filter		10pcs	
10. Sand recyling bottle		1pc	
11. New sand bottle		1pc	
12. Seal ring for sand bottle (67.9*3.7mm)	4pc		
13. Seal ring (17*1.5mm)		2pc	
14. Crystal sand		2bags	
15. Vacuum sand steel tube		1pc.	

C a t a l o g u e

General introduction.....	1
Parts narration.....	2
Characteristic.....	4
Effects.....	4
Operation.....	4
Nurse flow.....	5
Period of treatment.....	5
Response after treatment.....	5
After nurse.....	6
Notices.....	6
Applicable symptom.....	6
Contraindication.....	7
Accessories list.....	7
Technique parameter.....	7

Technique parameter

Voltage: AC220V/50Hz ☐ AC110V/60Hz ☐

Power: ≤150VA

Vacuum: ≤650mmHg

I General introduction

It is the dream for every woman to own skin just like exquisitely carved crystal, but it is always not the reality. As time passed, the year engraves the trace on our face because of air pollution, too much working pressure and so on. And all sorts of flaw also appeared on our skin: splash, wrinkle, cross pore, aging horniness accumulated, tarnish complexion..... what's more, the scar, concave, caused by acne, which always trouble us.

To do skin dermabrasion is the best way to solve these problems, the traditional way applies the chemical method and laser method to do skin dermabrasion, although it has a good effect, it will cause side effects even sequela such as skin become red, black, complexion asymmetry, pigment changes and etc. And it needs a long time to recover which makes people worried.

At present, the most popular method to change skin—Microcrystal dermabrasion therapy. The microcrystal physics technology eliminates aging horniness and promotes new skin tissue to regenerate quickly, exactly and with no sting. It is famous for its safety, no side effect and remarkable curative effect.

T12 crystal & diamond dermabrasion instrument applies a kind of natural crystal powder which is sterilized through high temperature and will not bring chemical change, it makes effects through unique vacuum delivery system, it can remove aging cuticle of epidermis, stimulate capillary vessel expansion, improve skin rebirth ability and skin nature, reduce

IX After nurse

It is very important to choose the skin care product after the treatment, the product should be according to skin nature and symptom and then it will improve skin problem effectively.

The skin is frail during the treatment, so do not use irritative acidic product 3 days after the treatment, please wipe sun screen product, bring sunshade with you or put on your sunbonnet when you

X Notices

- (1) The skin should have no wound, tetter, inflammation or any symptoms before the dermabrasion treatment.
- (2) Keep the crystal powder from the eyes using the wet cotton pieces during the treatment.
- (3) Take good care of skin after the dermabrasion treatment, gentle and no irritative skin care product, and pay attention to anti-sunshine.
- (4) Do not shave or use horniness removal facial scrub when you do skin dermabrasion just for 3 days.
- (5) Do not stay at places with high temperature or high heat, lampblack, vapor bath, sauna and hot spring.

Applicable symptom

- ✧ The cutis aging horniness, big pore, coarse skin, lackluster, uneven Pigment, aging and flab, lack of elasticity, dark cycle.
- ✧ The middle cortex pore, acne, lipid granule, which, pockmark, beehive tissue inflammation, fleck (sunburn and other shallow pigmentary spot)
- ✧ Deep cortex pregnancy lines, shallow scar, concavo-convex holes.

4. If you would like to use diamond dermabrasion rod, Please unplug black hard tube A of the sand dermabrasion handle from [4], Insert the tube of diamond

VI Nurse steps

- 1 Clean skin, remove skin grease and dust
- 2 Clean skin with warm water and wait till no water remained on skin.
- 3 Sterilize hands and the crystal dermabrasion rod with 75% alcohol
- 4 Turn on the power switch [8], adjust output vacuum suction [2], regulate output sand quantities [1].
- 5 During operation, the beautician fasten one hand to skin; use the other to move the working rod on skin come-and-go slightly, with fingers swimmingly moved following the working rod.
- 6 The suction should adjust according to customers' endurance and in line with different skin nature and body parts.
- 7 Clean skin when the dermabrasion nurse completed.
- 8 Do nutrition ingredient injection with ultrasonic function.
- 9 To apply facial mask.
- 10 Wipe moisturizer and sun screen products when the facial mask Procedure completed.

VII Suggestion for period of treatment


6~8 times is a period of treatment, once every 2~3 week, 15~30 minutes for each time.

There is no limit for the microcrystal dermabrasion treatment; the time is mainly depended on patients' skin status, different body parts and the


VIII Response after treatment

There may be one-off florid or little feel like dry and tight when the dermabrasion treatment being done, but it doesn't matter, just wipe the skin moisturizer products will make it OK, and it is OK if

II. Parts narration

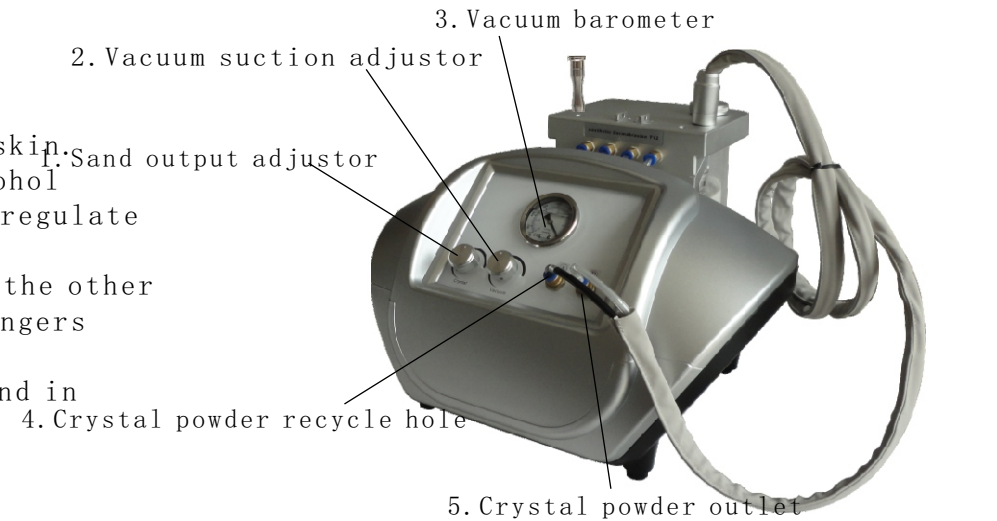

(Main body obverse)

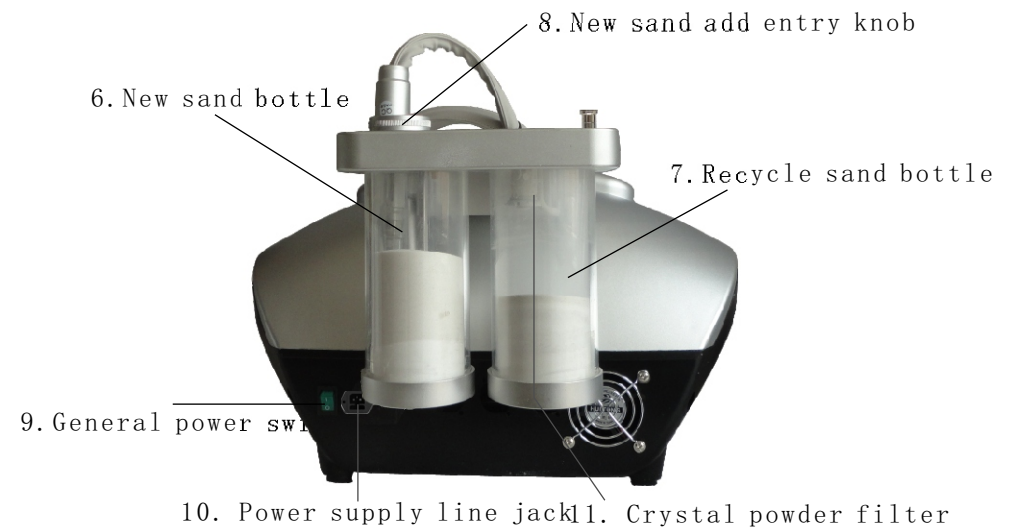

(Main body reverse)



Sand dermabrasion handle



Crystal dermabrasion cap



Diamond dermabrasion rod



Dermabrasion rod



Dermabrasion head

III Characteristic

1. Has the advantage of safe, simple and effective.
2. It is different from the traditional skin dermabrasion, applies the gradual method to do dermabrasion nurse, with no strong feel of pain, no need for anaesthesia, safe and reliable. with low risk
3. The crystal powder quantity and the vacuum pressure adjust are both controlled separately, which make it easy to

IV Effects

1. The strength created by microcrystal dermabrasion nurse can remove the aging horniness, promote epidermis renewal, stimulate regeneracy enginery and collagen protein hyperplasia.
2. The microcrystal dermabrasion stimulate blood circulation and lymph circulation in virtue of the vacuum suction, which can enhance skin Cells metabolism, activate deep-laid tissue and help skin to heal up.
3. The vacuum pressure suction removes the horniness, acne, oil and dirt of follicle, decreases acne bacilli multiply, acne

V Operation

Preparation before nurse

1. Insert black hard tube A of the sand dermabrasion handle to [4], insert transparent hard tube B to [5]
2. When the recycle sand bottle [7] is full, unscrew it in a clockwise direction, throwing away waste sand and fasten it in anticlockwise direction, please notice the sealing ring.