

# SPA7.0

## User manual



## **Warning**

Dear user:

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:

- ◆◆ Please read this instruction in detail, and keep it well for further reading and consulting.
- ◆◆ Please follow the struction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.




## Catalogue



<b>Parts instruction</b>	<b>1</b>
<b>General introduction</b>	<b>2</b>
<b>Operation</b>	<b>2</b>
<b>Caution</b>	<b>2</b>
<b>Forbidden group</b>	<b>3</b>
<b>Technical parameters</b>	<b>3</b>
<b>Annex List</b>	<b>3</b>

## Parts instruction

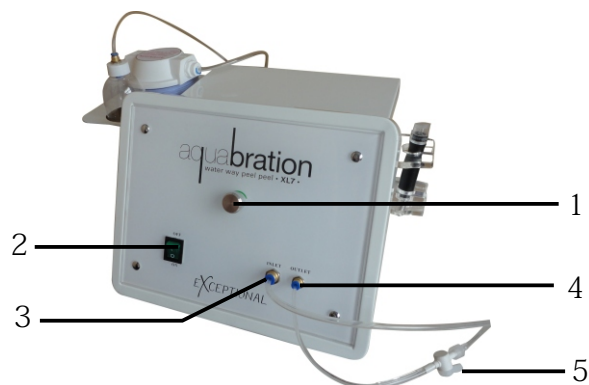

- ( 1 ) Vacuum debugging switch
- ( 2 ) General power switch
- ( 3 ) Vacuum Spa intake handle
- ( 4 ) Vacuum Spa outlet handle
- ( 5 ) Water regulating valve
- ( 6 ) liquid bottles intake handle
- ( 7 ) Intake handle for wastewater recycling bottle
- ( 8 ) Outlet handle for wastewater recycling bottle
- ( 9 ) Power supply line

## Bill of accessory



Vacuum Spa handle




Liquid bottle



Wastewater recycling bottle

### Vacuum Spa heads

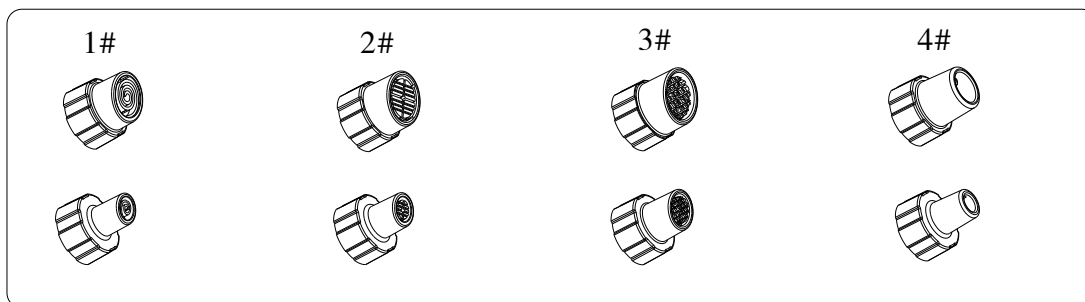

## General introduction

SPA Skin beauty machine completely changed the traditional way, which is hand-cleaning skin relying on individual's practice skills, SPA Skin beauty machine, using vacuum suction mode controlled by intelligent process, through the combination of products and equipment, deep cleaning the skin and pores off horny, acne, blackheads and other impurities in very short period of time. And improve the deep absorption of nutrition products, promote pores tightening, smooth skin, increase skin moisture, and make your skin whitening, moisturizing and good texture.

## Operation

1. Install the annex on the host, place all the annex and connect with the host:

(1) Insert the vacuum Spa intake handle into [3]

(2) Insert the vacuum Spa outlet handle (with Water regulating valve) into [4].

(3) Install the liquid bottles intake handle into [6].

(4) Install the wastewater bottles intake handle into [7].

(5) Install the wastewater bottles outlet handle into [8].

(6) Install the Power supply line into [9].

2. Put appropriate liquid into the bottle

3. Choose the vacuum work head, and connect to the vacuum spa handle.

1# work head for clearing comedones, Whitehead and acne

2# work head for deep cleaning of normal skin, uneven color and dark skin, dry aging skin

3# work head for thick cuticle, coarse pores, oil over secretion skin.

4. Open the power switch [2].

5. Adjust the vacuum debugging switch [1] (left – Small, right – large), transferred into a suitable pressure.

6. Adjust the Water regulating valve [5], transferred into the appropriate amount of water. Beautician hold the vacuum spa handle on skin for a while until the water flow into the work head, and then drop some water on skin by points, to make the skin surface wet, and then apply cleansing product on the skin, and move the vacuum work head on skin back and forth.

## Steps

Dermabrasion: Deep cleaning, clear skin stratum corneum, clearing blackhead.

## Caution

1. Do not use on wound.
2. Pour out the water in the waste recycling bottle in time, to avoid damage caused by overflowing.
3. Different size of vacuum work head can be flush with water directly, then soaked with 75% Alcohol.

## Forbidden group

1. People who has fever, infectious diseases, acute diseases.
2. people with hemorrhagic disease, trauma, vascular rupture, skin inflammation, skin disease.

## Technical parameters

Input Voltage: AC/220V 50Hz

Power: < 80W

Output pressure: 100KPa

## Annex List

1. Host	1pc
2. Vacuum Spa handle	1pc
3. Vacuum Spa work head	2pc(16pcs)
4. 500ml liquid bottles	1pc
5. Wastewater recycling bottle	1Sets
6. Power supply line	1pcs
7. Accessories shelf	2pc
8. User manual	1pcs
9. Hexagonal screws	6pcs
10. Hexagon screwdriver	1pcs
11. Seal Circle of vacuum Spa handle 1.2*2mm	2pcs