


Air Pressure Body Slimming Suit



Air Pressure Body Slimming Suit


Air bag:

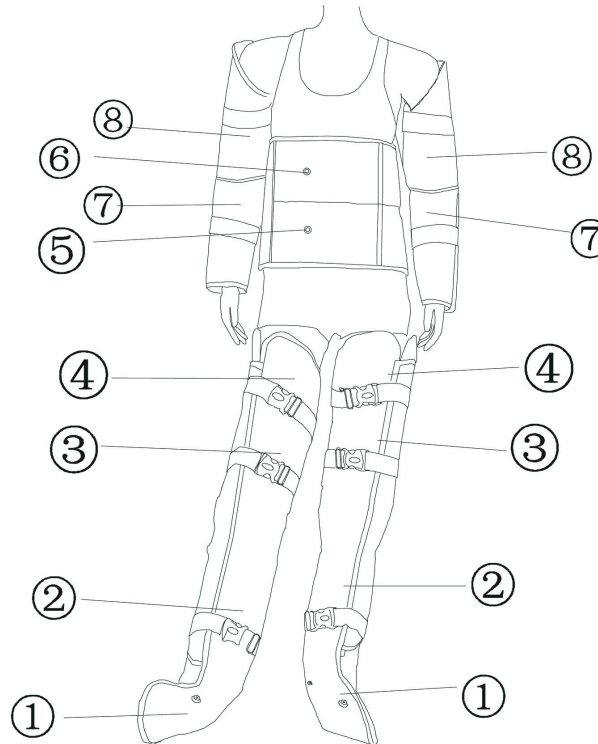

Function:

1. Detoxify;
2. Body shapes; Tone body muscles;
3. Body slimming;
4. Physical treatment for lymphatic;
5. Relaxation;
6. Cellulites reducing.

Control Panel:

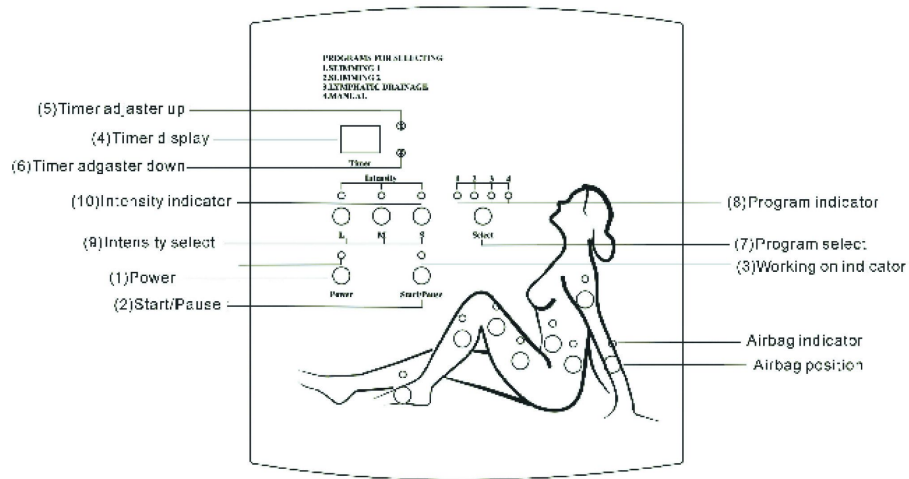

Operation:

1. Turn the POWER (1) , the indicator will be on.
2. Adjust the timer (5) , the pre-set is 15 minutes
3. Press SELECT (7) , the indicator (8) will be on

3 Auto-control program:

- 1-Slimming 1
- 2-Slimming 2
- 3-Lymphatic

After choose the program, then press START/PAUSE button (2). The airbag will work one by one under the program setting and the indicator will be on when working. The direction of air pressure is form down to up.

Manual program:

4-for manual usage: after press 4, you can choose and press any airbag, which you want it to act. For example, you only want to use the waist airbag, please press waist button in the control panel, the airbag will be ready to act. After choosing the airbag, then press START/PAUSE button (2). The airbag will worked one by one under your order and the indicator will be on when working.

4. During using, you can set the intensity through (9) to select L, M and S.
L-large
M- medium
S-small
5. Press the START/PAUSE button (2) to start and hold the program.

Treatment:

- Fat : select 1 program, standard term is 60minutes and choose the “L” for the intensity.
- Medium fat body: select 1 or 2 program, standard term is 30-45 minutes and choose the “L or M” for the intensity.
- A little fat body : Medium fat body: select 2 program, standard term is 30-45 minutes and choose the “M or S” for the intensity.
- Keep weight: select manual program and choose 5 airbags which the customer want to keep shape, and standard term is 20~30minutes
- lymphatic: select 3 program, standard term is 20~35minutes.

Cautions:

1. Client has to take off all jewelry before treatment.
2. Client with metal inside his/her body and pregnant women are not recommended to use this machine.
3. Turn off and unplug machine after one day's usage. Just use dry clothe to wipe the machine.