

Weight-losing Master

INSTRUCTION




Dear customer:
Thanks for your choosing our products.
For the best capability and safety, please read the manual before operation.
Please preserve the manual for your reference.

PRODUCT FUNCTIONS

This apparatus was carefully designed according to human bionice research, which combines functions of broken-fat, fat dissolved, lymphatic drainage, magnetic therapy of firm skin in one and using high-tech means to massage, do electrotherapy and pressure therapy to get significant effect.

Slimming

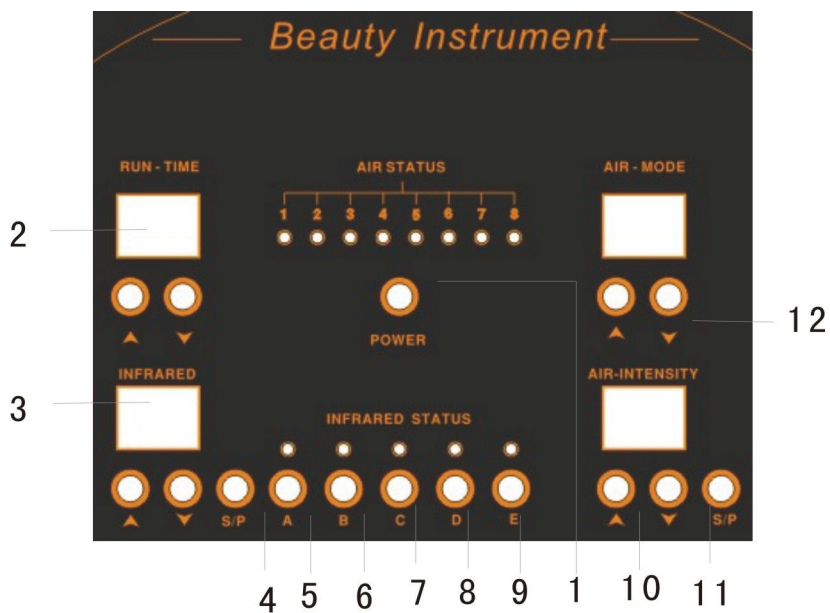


1. boot-strap
2. time
3. heat energy intensity function
4. heat energy to start/suspend
5. left hand heat energy to select
6. right hand heat energy to select
7. waist heat energy to select
8. left foot heat energy to select
9. right foot heat energy to select

- 10. charge intensity function
- 11. charge function to start/suspend
- 12. charge mode

CONTROL SCREEN

Accessories Connect Diagram

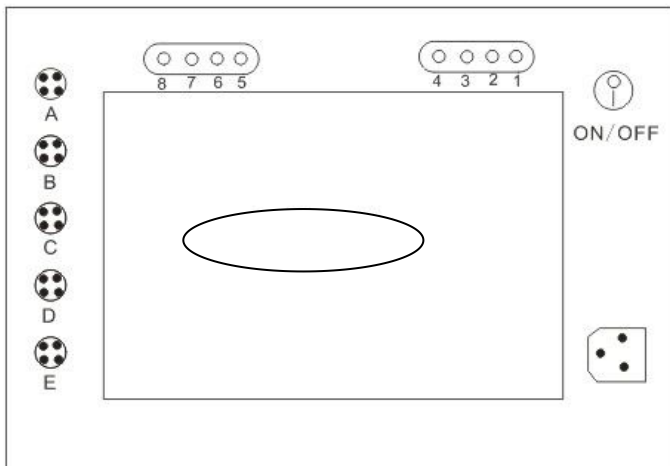

Rear Cover of Chassis


Output of air pressure de-toxin

Output of spare part for foot FOOT 0 FOOT 1

Output of spare part for leg LEG2 LEG3 LEG 4 LEG5

Output of spare part for waist WAIST6 WAIST7


Output of spare part for arm ARM8 ARM9

	Spare part of arm Air pressure ouput 8/9 Infrared heating output 1.2	
	Spare part of waist Air pressure ouput6/7 Infrared heating output 3	
	Spare part of leg Air pressure ouput2/3/4/5 Infrared heating output4/5	
	Blanket	
	Spare part of boot Air pressure output 0/1	
	Infrared heating connector	
	Air pressure pipe Connected with output	

		Air pressure connector 9 Air pressure connector 8 Infrared heating connector 1 Infrared heating connector 2 Air pressure connector 7
		Air pressure connector 6
		Infrared heating connector 3 Infrared heating connector 4 Infrared heating connector 5 Air pressure connector 5 Air pressure connector 4
		Air pressure connector 3 Air pressure connector 2
		Air pressure connector 1
		Infrared heating connector 4 Infrared heating connector 5
		Air pressure connector 0

Please connect the pipe with the correct number


Air pressure connect diagram

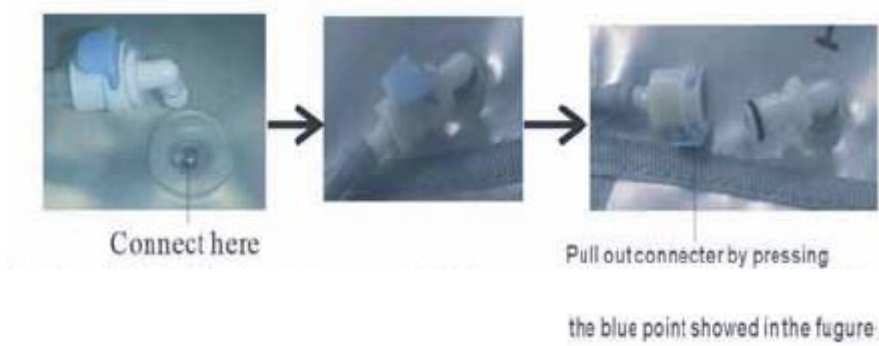

Infrared heating connect diagram



Connect Diagram

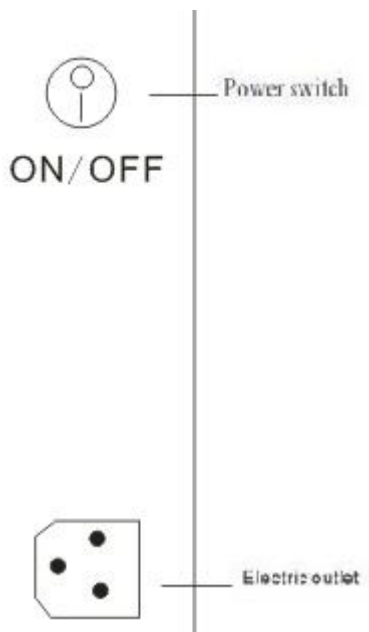

Electric Outlet

Fuse

Power Switch

OPERATION INSTRUCTION

1. Check and connect the power wire.
2. Turn on the power, the machine begins to work.
3. Connect the output port to the slimming clothes. To make sure, the display is shown as "0"
4. Two methods to wear the slimming clothes and electrode patches:
 Put the electrode patches to the parts, with which you are not satisfied, and put on the slimming clothes for waist, arms and legs.
 Please wrap the preservative film and then put on the slimming clothes.
5. Press the POWER button on the control panel.
6. Adjust Time: According to the working time to adjust the time on the machine of Button 10. The minimum time is one minute and the maximum is 60 minutes. Usually, the suitable time is 30 to 60 minutes.
7. Far Infrared
 Far Infrared Adjust: Press Button 20 and 21 to adjust the intensity of far infrared. There are eight levels. According to the request of client to press "▼" and "▲" and select a proper intensity.
 Note: Because the skin nature is different for everyone, please don't choose high intensity. To avoid burning, please wrap wet towel before put on the slimming clothes. It's shown as the machine is working while the little red spot sparkles under the right of the Liquid Crystal Display.
8. Air-pressure De-toxin Mode Adjust: There are three modes for air pressure; Mode I, from "1" to "8"; Mode II, from "8" to "1"; Mode III, "1" and "2" first, then "2" and "3" followed with "7" and "8"; Mode IV, "7" and

“8” first, then “6” and “7” followed with “1” and “2”; Mode V, “1” and “8” first, then “2” and “7” followed with “4” and “5”; Mode VI, “4” and “5” first, then “3” and “6” followed with “1” and “8”.

TABOO

The following people are prohibited from the instrument treatment

BASIC GUIDE FOR MASSAGE



1. Press skin surface



2. Massage method around the knee



3. Deep-press method



4. Wring method



5. Pounding method



6. Chopping method



7. Wring method with two hands




8. Grasping method from center part to side on back and belly




9. Press on back

4. Air pressure massage for 15—30 minutes.

(1) Please wrap the client with preservative film on the body they want treatment. Then the client should put on the slimming cloth accessory and lie on the facial bed.

(2) Start the machine by pressing the power button. 

(4) Press the button of output start/stop. 


(


OTHER PROPOSAL FOR AIR PRESSURE

OTHER PROPOSAL FOR AIR PRESSURE

1. Lose weight for legs

(i) Put on the slimming cloth accessory of boots and legs


(ii) Press the button 

(3) Adjust the air pressure mode and time as  (Look at the panel diagram 17,18,19,20)

3. Air pressure de-toxin

(1) Put on the slimming cloth

(2) Press buttons 

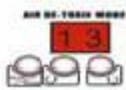(3) Adjust the air pressure mode and frequency as 



SPECIFICATIONS

POWER SUPPLY: AC

ACCESSORIES PACKING LIST

Item Name	Unit	Quantity	Remarks
Mainframe	Set	1	
Slimming cloth	Set	1	
Elastic band	Set	1	
Blanket	Set	1	
Power wire	Strip	1	
Fuse	Piece	2	
Instruction	Piece	1	

ORDINARY MALFUNCTION REMOVING

ORDINARY MALFUNCTION REMOVING

Malfunction	Removing
No work when starting	Checking power if been well connected
	Checking power if been turned on
Slimming cloth no heating	Checking slimming cloth if been well connected with machine
Slimming cloth no inflation	Checking plastic connecter if been well connected with machine
When working machine overheating	Over working, please turn off machine to coolingoff
	Checking the AC if normal (checking by professionals)

IMPORTANT SAFETY NOTICE

- 1. Please don't disassemble the equipment or try to do some other operation, which have not been indicated in our instruction. All the repairing work should be done by our admitted professional personnel.**
- 2. Please don't assemble or operate this equipment while you are near the water or your hands are wet. Notice that don't pour the liquid on it.**
3. Please pull out the plug from the socket when you met such conditions as below. And keep in touch with professional maintenance Station.
 - 1> The equipment touched liquid
 - 2> The equipment has abnormal smell, smoking or loud and strong noise.
 - 3> The Cable is broken
 - 4> The equipment is fallen down and broken
4. Please don't put anything on the cables, don't place the equipment at any place where someone can step on the cable.
5. Please pull out the power cable after using it for your safety.
6. Don't put anything into the equipment's jack, otherwise it may cause fire or touch the electricity. If something enters the equipment, please don't pick it out by yourself, contract distributor or our company immediately.
7. Please don't put the equipment on the anything that can't stand steadily, avoid it turns over and damage.
8. Our company has the right to modify the information in this introduction before notify! And keep the explanations right for the above.